

**MedTrak VNG
FallTrak II Balance System (504024)
User's Guide**

*Copyright ©2015 MedTrak VNG
Henderson, NV
All rights reserved*

Disclaimer:

MedTrak VNG makes no representations or warranties regarding the contents of this manual. We reserve the right to revise this document at any time or to make changes to the product described within it without notice or obligation to notify any person of such revisions or changes.

Printed in the United States of America

Part Nbr: 300921 Rev: C



IMPORTANT INFORMATION

Qualified Personnel Notice

WARNING: The FallTrak II Balance Systems contain no user serviceable parts. Repair of the FallTrak II System should only be performed by qualified service personnel.

Classification Information

- EMC Certified to: EN 60601-1-2.
- Class: Type (BF) equipment.
- CE Conformity to MDD 93/42/EEC.

Cleaning and Maintenance

CAUTION: Do not use Benzene, thinner, or any volatile substance to clean the unit as they may leave a permanent mark.

Balance Platform

- With the system turned OFF, use a lint free cloth dampened with a mild detergent or 70% isopropyl alcohol to clean all outer surfaces.
- The user is responsible for disposal of any FallTrak II components per local regulations.

Foam Pad

- Antimicrobial coated positioners and table pads are highly resistant to fluid infiltration. In order to maintain the original integrity of the coating, you should exercise care in their handling and use. Wipe your antimicrobial coated products clean as soon as possible after fluid contact or use. A soft cloth with 70% isopropyl alcohol or 10% bleach / water solution is recommended. An all-purpose cleaner similar to 409™ or Windex™ may also be used. Allow antimicrobial coated products to air dry in a flat (natural) position and store them in a flat (natural) position at a temperature not to exceed 170° F.

Assembly and Installation Notes

- The FallTrak II System can be installed in any Ordinary Location.
- The FallTrak II Balance system should be placed on a hard, flat and level surface.

Components

- FallTrak II Balance Board with USB Cable.
- FallTrak II Software on DVD
- FallTrak II Balance User Guide

Available options

- Soft Surface with Grid

Specifications

Balance Board

Parameter	Value
Size	18" L x 10.5" W x 2.25" H.
Weight	8.25 lbs.
Power	USB-powered, 5v, 300 ma.
Patient weight capacity	330 lb. (150 kg).

Ratings

- The FallTrak II System is not intended for use in the presence of flammable anesthetics.

- The FallTrak II System is rated for Continuous Operation.
- Operating Temperature 10° Centigrade to 35° Centigrade.
- Storage Temperature -10° Centigrade to 55° Centigrade.
- Operating Relative Humidity 20% noncondensing to 80% noncondensing.
- Storage Relative Humidity 20% noncondensing to 80% noncondensing.
- Special cooling is not required for the operation of the FallTrak II.
- Ordinary protection against ingress of waters must be provided.

Accessories

- The FallTrak II must be operated with an ITE (Information Technology Equipment) compliant PC.
- Only MedTrak VNG approve accessories may be used with the FallTrak II System.

Symbols

Symbol	Definition
	Caution
	WEES directive
	Temperature Limits
	Humidity Limits
	DC Voltage
	Type BF Applied Part
	Class II Equipment

Additional Notes/Warnings

Symbol	Definition
	The user is responsible for disposal of the system in compliance with local regulations.
	Note: It is not recommended that the FallTrak II System be changed in any way. If any changes are made, adherence to the CE standards becomes the owner's responsibility, if you have any questions, please contact the CSMI Customer Services Department at: (Voice) 347.742.4100 or (FAX) 718.228.7797 or (e-mail) scott@medtrakvng.com .



CAUTION: Federal law restricts this device to sale of or on the order of a medical practitioner. When prescribed for therapeutic purpose, a physician should clearly define the parameters of use (i.e., total work, maximum heart rate, etc.) to reduce the risk of patient injury.

Copyrights and Trademarks

The software used to operate either system is protected under copyright laws. Any use of the software other than its intended use with the FallTrak II Balance system is prohibited. Altering or tampering of the software in any manner constitutes an unwarranted use of the system and immediately voids all warranties expressed or implied by MedTrak VNG. MedTrak VNG assumes no liability for damaged equipment or harm to any individual as a result of malfunction due to tampered software.

FallTrak IIMicrosoft is a trademark and Windows is a trademark of Microsoft Corporation.

IMPORTANT PRE-USE CONSIDERATIONS

Read carefully BEFORE operating the FallTrak II Balance System

The FallTrak II System provides a wide variety of choices for overall treatment of your patients. Adhere to the following precautions and instructions to ensure optimum patient safety and proper system usage. DO NOT operate the system until you fully understand its operation and have either:

- attended the in-service provided as a service by CSMI, or
- fully read and understand the documentation provided with the system.

NOTE: Federal law restricts the use of this device to, or on the order of, a licensed physician or licensed practitioner. If you have any questions, please call the MedTrak VNG Technical Services Department at: 347.742.4100.

When using the System, It is the operator's responsibility to:

- take the time to properly instruct all patients especially those who have difficulty following and carrying out instructions. It is highly recommended that such patients be continually coached and verbally guided throughout the session;
- closely monitor all patients using the system;
- ensure that a clinician is ready to assist in case of loss of balance when patients are working with their eyes closed;
- ensure the patient is standing in the center of the platform for optimal operation,
- position the display so that the patient can look straight at it. This will help ensure good posture during the test or exercise session.

Additional instructions:

- A walker can be used as needed for patient's that require or feel more comfortable with something to hold onto if needed. **Note:** It will be important, however, to indicate a manual support was required on the test output.
- There is a learning curve that must be considered when testing with this device. Clinical research suggests practice trials be performed prior to testing.
- It is highly recommended that the clinician remain with the patient during testing or training.

IMPORTANT: The FallTrak II Balance system provides some compliance to adjust to non-level flooring. Leaning fully over the side of the platform can result in the platform tilting up to 3 degrees.

TABLE OF CONTENTS

IMPORTANT INFORMATION	2
Qualified Personnel Notice	2
Classification Information	2
Cleaning and Maintenance	2
Assembly and Installation Notes	2
Components	2
Available options	2
Specifications	2
Ratings	2
Accessories	3
Symbols	3
Additional Notes/Warnings	3
Copyrights and Trademarks	4
IMPORTANT PRE-USE CONSIDERATIONS	5
FALLTRAK II FallTrak IIINSTALLATION AND SETUP	11
Minimum System Requirements	11
Package Contents	11
Installation Procedure	11
Overview	12
Feedback Displays	13
CTSIB (Force Plate Mode)	13
Stability (Force Plate Mode or Tilt Mode)	16
Mobility (Force Plate Mode or Tilt Mode)	17
Stability Envelope (Force Plate Mode or Tilt Mode)	18
Weight Bearing (Force Plate Mode)	20
Weight Bearing XY (Force Plate Mode)	22
Weight Shift (Force Platform Mode and Tilt Mode)	23
Center of Pressure (COP) (Force Plate Mode or Tilt Mode)	25
Limits of Stability (LOS) (Force Plate Mode or Tilt Mode)	26
Targets (Force Plate Mode or Tilt Mode)	28
Random Motion (Force Plate Mode or Tilt Mode)	30
Force vs. Time (Force Plate Mode)	32
Scale (Force Plate Mode)	34
Tilt (Force Plate Mode or Tilt Mode)	36
Roadway (Force Plate Mode or Tilt Mode)	37

Pong (Force Plate Mode or Tilt Mode)	40
Breakout (Force Plate Mode or Tilt Mode).....	42
Balance Board (Force Plate Mode or Tilt Mode)	44
Skier (Force Plate Mode or Tilt Mode).....	45
Snowboarder (Force Plate Mode or Tilt Mode).....	46
Pilot (Force Plate Mode or Tilt Mode)	48
The Main FallTrak II Screen.....	49
Starting FallTrak II Program	51
Testing a Patient	52
Patient Selection.....	53
Adding a New Patient	54
Pattern Selection	56
Protocol Selection.....	57
Test Status	58
Patient Setup	59
Anatomical Zero.....	60
Move to COP Timer	61
Patient Feedback (CTSIB Test)	62
Test Status	63
Steps	63
REPORTS	64
Single Test Report.....	64
Progress Report	66
Involved Side Re-Test Report.....	67
DASHBOARD	68
GROUP SUMMARY.....	69
Computing a Group Summary	69
Adding a New Group Summary to the Database.....	69
Computing the Summary.....	69
Summarizing the Data	70
Exporting a Summary to Microsoft Access	71
Printing a Summary Report	71
Comparing a Patient to a Group Summary.....	71
Deleting a Group Summary	71
UTILITIES	73
Edit Background Information.....	73
Deleting Stored Tests.....	74

Usage Report	75
Backup	76
Data File Location	77
Compress	78
Import: Database	79
Import: Protocols.....	80
Exporting Data	81
Submitting a Test to MedTrak VNG	82
System Protocols	83
Interactive Protocols.....	84
Protocol Editor: Roadway	84
FallTrak II Data File Locations	85
Users	86
PREFERENCES.....	88
Language.....	88
Touch Screen Display.....	88
FallTrak II Interface Type	88
Use Sound to Count Repetitions.....	88
Reset Torque Scales for Each Set.....	89
Report Heading.....	89
Units.....	89
Limits of Stability Normative Values	89
Stability Hold Time.....	89
Stability Goal.....	90
Weight Bearing Display.....	90
Measured Parameters	91
Sensor Layout	91
Patient Center of Mass (Force Platform Mode).....	91
Weight (Force Platform Mode).....	91
Center of Pressure – Balance Platform (Force Platform Mode)	91
Foot Position.....	92
Center of Pressure - Patient (Force Platform Mode)	92
COP Length, Angle, Direction (Force Platform Mode)	93
Displayed Tilt Angle, Tilt Direction (Force Platform Mode)	93
Displayed R/L and A/P Tilt Angle (Tilt Mode).....	93
Percent Weight Bearing (Force Platform Mode)	94
Limits of Stability Score (Force Platform Mode and Tilt Mode)	94

Stability Score (Force Platform Mode and Tilt Mode)	94
Mobility Score (Force Platform Mode and Tilt Mode).....	95
CTSIB Score (Force Platform Mode)	95
Excel Export Fields	95
ELECTROMAGNETIC COMPATIBILITY	97
Table 1	97
Table 2	98
Table 3	99
Table 4	100

LIST OF FIGURES

Figure 1 Hit	24
Figure 2 Miss.....	24
Figure 3 Wrong Direction	24
Figure 4 Force in Target.....	32
Figure 5 Force Out of Target.....	32
Figure 6 FallTrak II Form	49
Figure 7 Title Bar.....	49
Figure 8 Menu Bar	49
Figure 10 Button Bar No Patient Selected	50
Figure 10 Button Bar Patient Selected.....	50
Figure 12 Real-Time Display	50
Figure 13 Status Bar.....	50
Figure 14 Select Machine Type.....	51
Figure 15 Select USB Port	51
Figure 16 Warm-up Screen	51
Figure 17 Patient Selection.....	53
Figure 18 Patient Information - General.....	54
Figure 19 Patient Information – Injury/Group.....	54
Figure 20 Pattern Selection	56
Figure 21 Protocol Selection.....	57
Figure 22 Test Status	58
Figure 23 Patient Setup Screen	59
Figure 24 Set Anatomical Zero Screen	60
Figure 25 Move to COP Timer.....	61
Figure 26 CTSIB Feedback Display (COP)	62
Figure 27 Status Screen - Tests Completed	63
Figure 28 Reports Screen.....	64
Figure 29 Report: Actions	64
Figure 30 Report: Options	64
Figure 31 Progress Report Screen	66
Figure 32 Re-Test Report Screen	67
Figure 33 Dashboard Screen.....	68
Figure 34 Group Summary: Add	69
Figure 35 Group Summary: Filter	70
Figure 36 Group Summary: Select Tests.....	70
Figure 37 Group Summary: Side.....	70
Figure 38 Edit Background Information.....	73
Figure 39 Delete Stored Test	74
Figure 40 Usage Report	75
Figure 41 Backup Database	76
Figure 42 Import Database	79
Figure 43 System Protocol Editor	83
Figure 44 Protocol Editor: Roadway	84
Figure 45 Setup Users.....	86
Figure 46 Login	87

FALLTRAK II FallTrak II INSTALLATION AND SETUP

IMPORTANT: Install the FallTrak II software before connecting the FallTrak II Balance Platform to the PC.


Minimum System Requirements

Parameter	Value
Operating System	32 or 64-bit version of Windows XP Home or Pro, Windows Vista Home, Business or Ultimate, Windows7 Home, Premium or Ultimate.
CPU Type	Pentium or Equivalent
CPU Speed	1.00GHz
System RAM	1GB

Package Contents

Item
FallTrak II Balance Platform
FallTrak II DVD
User Manual (hard copy & pdf on DVD)

Installation Procedure

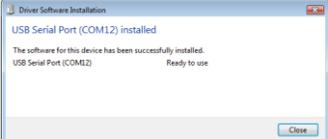STEP	DESCRIPTION	DETAIL
1	<p>Insert the FallTrak II CD into the CDROM drive of the PC/Laptop. The FallTrak II Installation Program should start automatically. If not select Start, My Computer and Double Click on the CD Drive that contains the FallTrak II Software.</p> <p>Select Next to accept the default installation settings until the software installation process is complete. The last button in the process will read Finish.</p>	
2	<p>With the FallTrak II CD still in the CDROM drive, plug the Balance Platform USB cable into the PC.</p>	
3	<p>Found New Hardware Wizard:</p> <ol style="list-style-type: none"> When the <i>“Connect to Windows Update”</i> dialog box appears, select No, not this time and click the Next button to continue. Continue clicking Next to accept the default installation settings until the software installation process is complete. If a message <i>“This software has not passed the Windows Logo testing”</i> appears, click the Continue Anyway button. <p>The New Hardware Wizard will run a second time. Select the same choices as listed in Steps A-C.</p>	
4	<p>The FallTrak II Balance Installation is now complete.</p>	

Overview

Although functional assessments provide the clinician with a general idea of who may be at risk, they do not identify impairments within specific body systems (e.g., motor, sensory) or why the problem exists. In addition, many of these types of assessment are not sensitive to changes that may result following intervention. The FallTrak II Balance System is a static force plate (Force Plate Mode) that can provide objective and quantifiable assessments of underlying impairments in the sensory and motor systems. The FallTrak II Balance & Tilt System adds a tilt sensor to the basic FallTrak II Balance System allowing both static force (Force Plate Mode) and dynamic tilt (Tilt Mode) measurements. These parameters are used in a variety of protocols to evaluate and improve the multiple dimensions of balance. This system can be used with a variety of different clinical populations (e.g., patients with orthopedic and/or neurologic disorders) in outpatient settings and with individuals experiencing postural instability in non-clinical settings (e.g., senior living communities, senior centers, fitness facilities). The training capabilities of the FallTrak II System also facilitate the reduction or elimination of impairments that contribute to poor balance and mobility. The real-time biofeedback available will also help your patients/clients perform the exercises correctly as a result of the moment-to-moment feedback provided.

This manual starts by providing an overview of each feedback display,, followed by the steps to run a CTSIB test, and finally a description of each measured parameter and how it is calculated.

Feedback Displays

CTSIB (Force Plate Mode)

Assessment of Sensory System Impairment Using The mCTSIB

Purpose of the Test

A modified version of the Clinical Test of Sensory Interaction in Balance (mCTSIB) (Shumway-Cook & Horak, 1986) is employed to evaluate the user's ability to use different sensory strategies (visual, vestibular, somatosensory) for maintaining upright balance. Although this test cannot distinguish between different sensory system impairments, it can be used to identify whether the use of sensory information in different sensory environments is normal or abnormal by comparing test results to age-based normative data.

The FallTrak II System's mCTSIB normative database includes the results of 261 tests administered to individuals aged between 12 and 88 years. Data is grouped in increments of 10 years (12 to 19, 20 to 29, 30 to 39, etc.). In addition, the FallTrak II Software's Group Summary program is capable of comparing normative data against the test data you collect.

Age	Male	Female	Total Subjects
12-19	11	15	26
20-29	10	18	28
30-39	12	11	23
40-49	10	13	23
50-59	10	19	29
60-69	17	39	56
70-79	20	75	65
80-89	4	7	11
Totals	94	167	261

Test Protocol

The full mCTSIB protocol tests the user in each of four different sensory conditions. It is up to the tester to determine which sensory conditions should be used to test the user. The following sensory systems are available to the user across the four sensory conditions:

- eyes open, stable surface (EOSS), Visual, Vestibular, Somatosensory;
- eyes closed, stable surface (ECSS), Vestibular, Somatosensory;
- eyes open, foam surface (EOFS), Visual, Vestibular; and
- eyes closed, foam surface (ECFS), Vestibular.

During each sensory condition, the user is required to stand upright and quietly for 10 seconds with feet shoulder width apart, arms folded across their chest, and eyes directed forward. The FallTrak II Screen is blank during the test so no visual feedback is provided. Trial time is provided on the screen along with an audible signal for the tester to start and stop each trial. During each trial, the FallTrak II records the user's center of pressure (COP). Based on COP data, the FallTrak II reports the following measured parameters:

Measured Parameter	Definition	Objective
Stability Score	Indication of stability	Reported as a percentage of maximum postural stability and compared to an age-based normative data value. The closer to 100% the better.

Path Length	The length of the COP trace.	The shorter the Path Length the more stable the user
Average Velocity	The average speed of the COP	The slower the average velocity the more stable the user

One trial for each condition is performed unless the person being tested is unable to maintain balance for the full duration of the trial. The test trial is immediately stopped if the person being tested:


- lifts his/her arms off the chest;
- prematurely opens his/her eyes on the eyes-closed conditions; or
- requires manual support due to a loss of balance.

If a trial is stopped it can be repeated. It is not recommended to ask a user to attempt a trial more than three times. The FallTrak II Software will only report the results from the last attempted trial. If more than one trial is attempted it should be noted on the test output. Repeating test trials allows the test administrator to determine whether the person being tested can “learn” to maintain balance if repeatedly exposed to the particular sensory environment.

Interpreting the Test Results

Increased sway or loss of balance observed in the eyes closed, stable surface condition indicates poor use of somatosensory inputs for balance, whereas increased sway or loss of balance in the two foam surface conditions may indicate poor use of visual-vestibular inputs (i.e., eyes open, foam surface) or vestibular inputs alone (i.e., eyes closed, foam surface). Although no cutoff scores have been developed for this test relative to fall risk, the information derived from each test condition can be used to determine whether the individual’s ability to use sensory inputs for maintaining upright balance is normal or abnormal. For example, abnormal use of sensory inputs would be indicated if an individual required two or more trials to successfully maintain balance for the allotted time of 10 seconds on any of the four test conditions, or was unable to complete the trials in any or all of the test conditions. It is important to understand that increased sway, particularly in the two foam-surface conditions, may also be due to lower-body muscle weakness.

Display

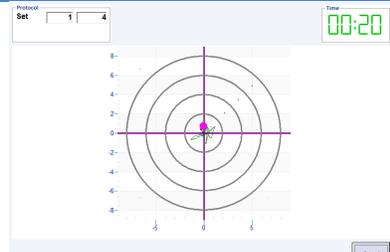
Display	Description
	<ul style="list-style-type: none"> • Provide a cursor (magenta ball) and tracing of the patient’s Center of Pressure. • A count-down timer.

Report

Report	Description
--------	-------------


Report

Description

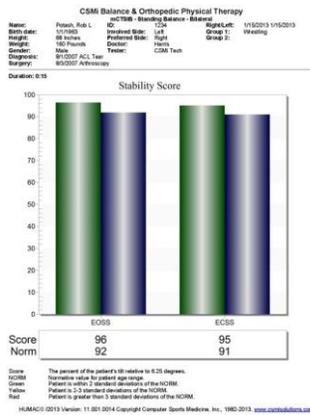

- Normative scores (Blue Bars) for the patient Age and Gender.
- Green Bar: Patient within 1 Standard Deviation of Normative Data.
- Yellow Bar: Patient within 2-3 Standard Deviations of Normative Data.
- Red Bar: Patient > 3 Standard Deviations of Normative Data.
- Score: The ratio of the patient's COP Std Dev to 6.25 degrees.
- NORM: The ratio of the normative group's COP Std Dev to 6.25 degrees.

File:///C:/Users/Public/Documents/CSM/HUMAC2013/Web/HUMACReport1.htm 3/23/2013

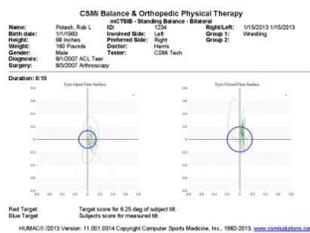

- A tracing of the patients real-time COP during the test.
- Blue circle: The Std Dev of the patient's COP.
- Red circle: The Std Dev of 6.25 degrees.

File:///C:/Users/Public/Documents/CSM/HUMAC2013/Web/HUMACReport1.htm 3/23/2013

Options

Termination

Display	Parameter	Definition
<input type="checkbox"/> Sets	Sets	Number of sets to perform
<input type="checkbox"/> Set Rest	Set Rest	Rest period (seconds) at the end of each set
<input type="checkbox"/> Time	Time	The duration (minutes:seconds) of each set.

Settings


Display	Parameter	Definition
<input checked="" type="checkbox"/> Eyes Open Firm Surface <input checked="" type="checkbox"/> Eyes Closed Firm Surface <input type="checkbox"/> Visual Conflict Firm Surface <input checked="" type="checkbox"/> Eyes Open Foam Surface <input checked="" type="checkbox"/> Eyes Closed Foam Surface <input type="checkbox"/> Visual Conflict Foam Surface	Settings	<ol style="list-style-type: none"> 1. Eyes open, firm surface 2. Eyes closed, firm surface 3. Visual Conflict, firm surface 4. Eyes closed, foam surface 5. Eyes closed, foam surface 6. Visual Conflict, foam surface

A CTSIB protocol typically uses conditions 1, 2, 4, and 5.


Stability (Force Plate Mode or Tilt Mode)

Stability test measures a patient's ability to stabilize their balance at locations around their neutral position. Results are reported as the percent of time a patient holds their COP in each of the eight targets.

Display

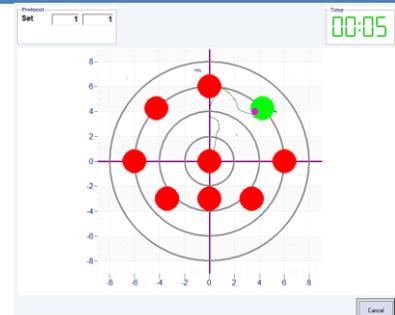Display	Description
	<ul style="list-style-type: none"> The current target flashes green when the patient is on-target and flashes yellow when they are off-target. A count-down timer displays the remaining time on the current target. The targets are selecting in a Clockwise order beginning with the target at 12:00.

Report

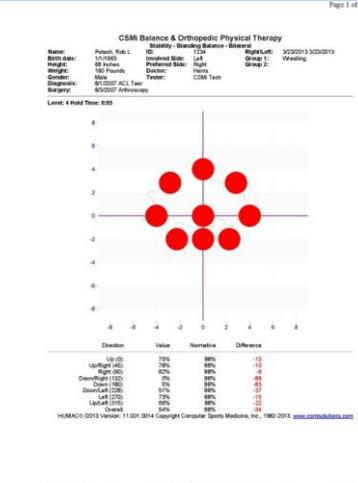Report	Description
	<ul style="list-style-type: none"> A plot of the targets and a tracing of the patient's COP during the test. The Score (percent of time) they were on-target. The Normative value (set with the File, Preferences command). The Difference between the Patient's Score and the Normative value.

Options

Termination

Display	Parameter	Definition
	Sets	Number of sets to perform
	Set Rest	Rest period (seconds) at the end of each set


Settings

Display	Parameter	Definition
	Level	Distance (degrees) to the center of the farthest target.
	Hold Time	Time in seconds the patient should remain on the target. Note: The FallTrak II counts-down the Hold Time so the patient knows when they should move to the next target.


Mobility (Force Plate Mode or Tilt Mode)

Mobility is used to test a patient's ability to hold their Center of Pressure on moving target that circles around a patient's neutral balance point. Results are reported as the percent of time a patient holds their COP inside the moving target.

Display

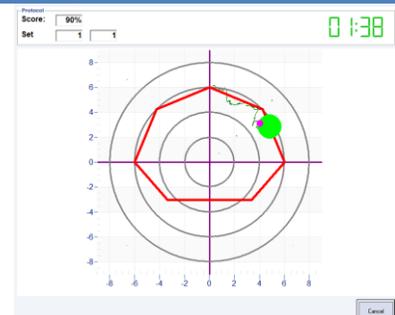Display	Description
	<ul style="list-style-type: none"> The current target flashes green when the patient is on-target and flashes yellow when they are off-target. A count-down timer displays the remaining time in the test.

Report

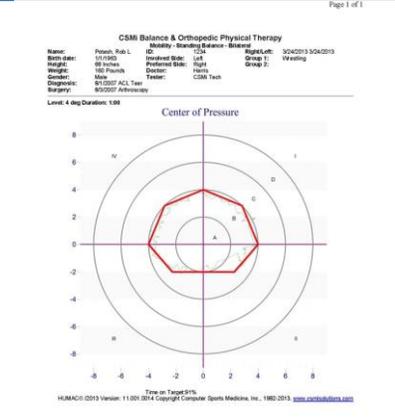Report	Description
	<ul style="list-style-type: none"> A plot of the target track and a tracing of the patient's COP during the test. The Score (percent of time) they were on-target.

Options

Termination

Display	Parameter	Definition
	Sets	Number of sets to perform
	Set Rest	Rest period (seconds) at the end of each set
	Time	The time (minutes:seconds) for the target to complete one revolution (one set).

Settings

Display	Parameter	Definition
	Level	Distance (degrees) to the center of the farthest target.

Stability Envelope (Force Plate Mode or Tilt Mode)

Assessment of Motor Impairments Using the Stability Envelope Test (SE)

The LOS Test described above provides the user with symmetrically spaced targets. If the user has an obvious deficit in one or more directions, the Stability Envelope Test may be more useful in documenting the user's region of stability and the type of postural strategy (i.e., ankle or hip) used to achieve maximal lean. The average individual can lean 8 degrees to the left, 8 degrees to the right, 8 degrees anterior, and 4 degrees posterior.

Purpose of the Test

This test measures how far individuals are able to lean through their region of stability in multiple directions and without altering the base of support (feet flat, no bending at hips, arms across chest).

Testing Protocol


The average individual can lean 8 degrees to the left, 8 degrees to the right, 8 degrees anterior, and 4 degrees posterior. The Stability Envelope asks the user to demonstrate how far they can lean in eight directions. When the user reaches a maximum degree of lean, the operator clicks on the user's on-screen COP to record the degree angle.

The test begins with the user's COP centered on the screen and an arrow indicating the direction the user should lean with feet maintaining balance, and arms folded across chest. The user leans in the direction of the arrow until they reach their maximum lean angle. The operator marks that point (by clicking the mouse or utilizing the touch screen), the point is recorded, and the arrow moves to the next position. The user leans in the direction of the arrow again (do not return to center). This test is completed when an octagon representing the user's Area of Lean is completed.

Interpreting the Test Results


The Stability Envelope Test measures the user's ability to lean (degrees) in each direction and their stability envelope (as defined by the area of the octagon). Normal lean angles are 8 degrees to the left, 8 degrees to the right, 8 degrees anterior, and 4 degrees posterior. This test documents the user's capabilities so the operator can investigate demonstrated inequities. Over time, improved angles of lean and increasing stability indicate improvement.

Display

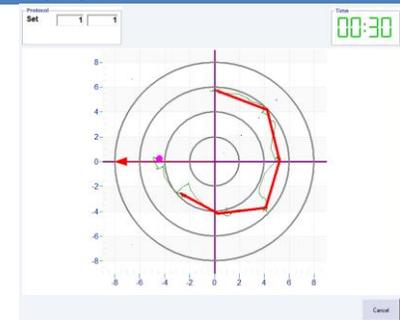Display	Description
	<ul style="list-style-type: none">• An arrow shows the current direction the patient should lean.• The magenta cursor shows the patient's location.• When the therapist clicks on the dot, the FallTrak II records the mouse position and progresses to the next location.• A count-up timer displays the total time.

Report

Report Description

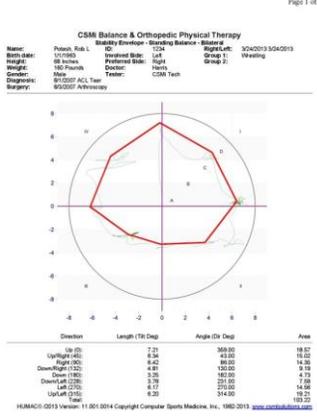

- A plot of the stability envelope (based on the locations recorded by the mouse clicks).
- A tracing of the patient’s COP during the test.
- The **Tilt Length** (degrees of patient tilt) in each direction.
- The **Tilt Angle** (orientation degrees)
- The **Area** of the section from the current direction to the next direction in the sequence.

Options

Termination

Display	Parameter	Definition
Sets Sets: <input type="text" value="1"/>	Sets	Number of sets to perform
Set Rest Set Rest: <input type="text" value="0:10"/>	Set Rest	Rest period (seconds) at the end of each set


Weight Bearing (Force Plate Mode)

Display the Patient's Right/Left or Anterior/Posterior weight distribution.

Display

Display	Description
	<p>Right/Left</p> <ul style="list-style-type: none"> • A pair of bars showing the patient's Right/Left Weight Distribution. • The Right/Left Percent or Weight value. • A count-down timer (if a Duration time is selected) or a count-up timer (if an Unlimited Duration is selected). <p>Anterior/Posterior</p> <ul style="list-style-type: none"> • A pair of bars showing the patient's Anterior/Posterior Weight Distribution. • The Anterior/Posterior Percent or Weight value. • A count-down timer (if a Duration time is selected) or a count-up timer (if an Unlimited Duration is selected).

Report

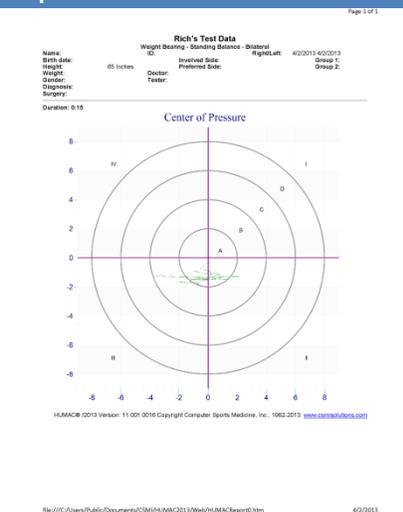Report	Description
	<ul style="list-style-type: none"> • A tracing of the patient's COP during the test.

Options

Termination

Display	Parameter	Definition
	Sets	Number of sets to perform
	Set Rest	Rest period (seconds) at the end of each set
	Time	The time (minutes:seconds) for the target to complete one revolution (one set).

Mode and Action

Display	Parameter	Definition
	>	Display the Patient's Right/Left weight bearing.
	∧	Display the Patient's Anterior/Posterior weight bearing.

Settings

Display	Parameter	Definition
	Display %	Display the percent R/L or A/P values below the feedback bars.
	Display Weight	Display the weight (pounds or KG) values below the feedback bars.

Weight Bearing XY (Force Plate Mode)

Display the Patient's Right/Left and Anterior/Posterior weight distribution.

Display

Display	Description
	<ul style="list-style-type: none"> • X and Y bars showing the patient's Right/Left and Anterior/Posterior Distribution. • The Percent value. • A count-down timer (if a Duration time is selected) or a count-up timer (if an Unlimited Duration is selected).

Report

Report	Description
	<ul style="list-style-type: none"> • A tracing of the patient's COP during the test.

Options


Termination

Display	Parameter	Definition
	Sets	Number of sets to perform
	Set Rest	Rest period (seconds) at the end of each set
	Time	The time (minutes:seconds) for the target to complete one revolution (one set).

Weight Shift (Force Platform Mode and Tilt Mode)

Weight Shift measures a patient's ability to shift their COP from right to left, anterior to posterior, or in between without going outside the target zone. Results measure good hit to bad hit ratio and time needed to complete the exercise.

Display

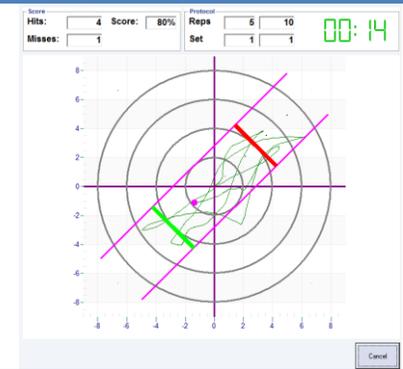
Display	Description
	<ul style="list-style-type: none">• The target for the patient to move through in green.• A Magenta Boundary the patient must stay within.• Their Score (Hits, Misses, Percent).• The number of Reps completed and the total number of Reps.• The number of Sets completed and the total number of Sets.• A count-up timer.

The goal is for the patient to move through the green maker in the correct direction while staying inside the magenta boundaries. If they stay within the boundaries a hit is scored (**Error! Reference source not found.**). If they move outside the boundaries a miss is scored (Figure 2). The patient must move through the marker in the correct direction for the system to move to the opposite marker. If the pass through in the wrong direction the same marker will stay green until they pass through in the correct direction (Figure 3). This would also be a miss since the patient went outside the magenta boundaries.


Center of Pressure (COP) (Force Plate Mode or Tilt Mode)

Center of Pressures is used to measure the variation in a patient's center of pressure during exercise. Display the Patient's Right/Left and Anterior/Posterior weight distribution.

Display

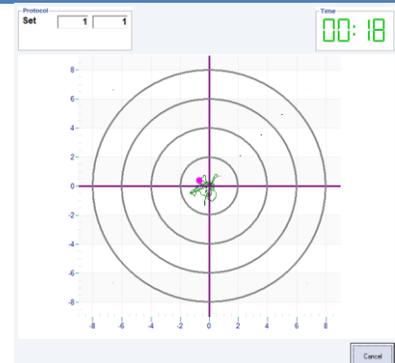
Display	Description
	<ul style="list-style-type: none"> • A trace of the patient's Center of Pressure. • A count-down timer (if a Duration time is selected) or a count-up timer (if an Unlimited Duration is selected).

Report

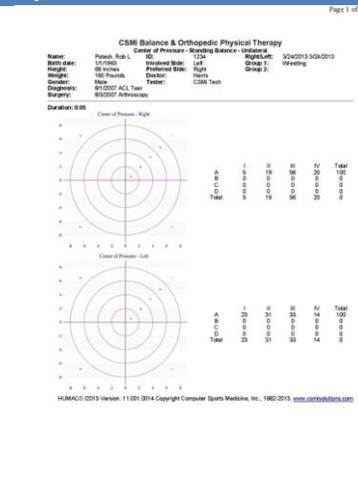Report	Description
	<ul style="list-style-type: none"> • A tracing of the patient's COP during the test. • The percent of time the patient spent in each Ring and each Quadrant.

Options

Termination

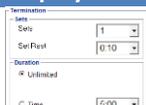Display	Parameter	Definition
	Sets	Number of sets to perform
	Set Rest	Rest period (seconds) at the end of each set
	Time	The time (minutes:seconds) for the target to complete one revolution (one set).

Limits of Stability (LOS) (Force Plate Mode or Tilt Mode)

Assessment of Motor Impairments Using the Limits of Stability Test (LOS)

Possible motor impairments related to the voluntary planning and execution of movements can be identified using the Limits of Stability Test. This test provides the operator with information about the size of each individual's region of stability and the type of postural strategy (i.e., ankle or hip) used to achieve maximal lean. From a functional perspective, the ability to voluntarily move the Center of Mass (COM) to positions within the limits of stability is fundamental to the successful performance of mobility tasks such as reaching for objects, transferring from a seated to standing position, and walking. The average individual can lean 8 degrees to the left, 8 degrees to the right, 8 degrees anterior, and 4 degrees posterior.

Purpose of the Test

This test measures how far individuals are able to lean through their region of stability in multiple directions without altering the base of support (feet flat, arms across chest). Also evaluated is how quickly and directly individuals are able to lean through their limits of stability.

Testing Protocol

The average individual can lean 8 degrees to the left, 8 degrees to the right, 8 degrees anterior, and 4 degrees posterior. The posterior target is also set to 1/2 the distance of the anterior target. "Degrees" refers to the extent that the patient is leaning with feet flat and no losing balance. The Limits of Stability Test begins with the user's COP icon on the center target (representing the neutral position) and eight additional targets circling the center target. The user is instructed to move the small pink circle icon, presenting their COP, away from the highlighted center and in the direction of the highlighted outer target by leaning as quickly and directly towards the target as possible. It will be important to instruct users to only lean as far as they safely can and without losing balance. This is repeated until all of the targets have been reached. If a user cannot reach a target, the tester must press the Skip button.

The tester can adjust the LOS protocol by degrees (distance between the center and perimeter targets) and by the amount of time the user is required to remain on the highlighted target in order to demonstrate control/stability. It is recommended that the tester use a protocol of 8 degrees. For training users can start with a small degree setting and increase the setting as they successfully completes a protocol. For example, begin with 3 degrees and with a 1-second hold; if completed successfully continue to 4 degrees with a 1-second hold until the user is not able to reach all of the perimeter targets. The FallTrak II Software allows you to build protocols with sets containing increasing levels of difficulty.

Interpreting the Test Results

The LOS Test measures the user's ability to lean through their stability limits (degrees). The FallTrak II reports a Stability Score (ratio between the distance the patient moved and a straight line) and average velocity. The inability to reach a target or a poor stability score relative to the other target scores could be an indication of limited Range-of-Motion, strength, or motor control in that direction.. A poor stability score in all directions may be an indication of poor neuromuscular control.


Display

Display	Description
---------	-------------


Targets (Force Plate Mode or Tilt Mode)

Place targets at specific locations which the patient must reach. After reaching each target, the patient must return to a neutral position.

Display

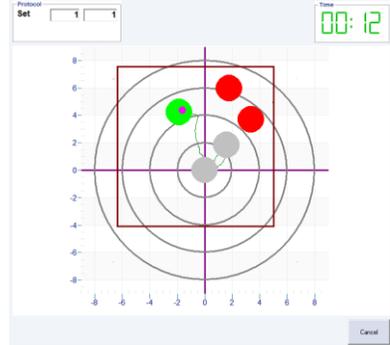
Display	Description
	<ul style="list-style-type: none"> Click on a blank area to add a target. Patients will be required to progress through the targets in the order they are added to the screen. Note: The patient's Range of Motion is displayed as a red boarder. Targets should be placed within this area. Click on an existing target to remove it from the screen. After all targets are placed, click Go to begin the test. The targets cannot be changed once the test begins. The current target flashes green when the patient is on-target and flashes yellow when they are off-target. A count-up timer displays the total test time. The targets are selecting in the order they are placed on the screen by the therapist. The patient must return to the center after each target is reached.

Report

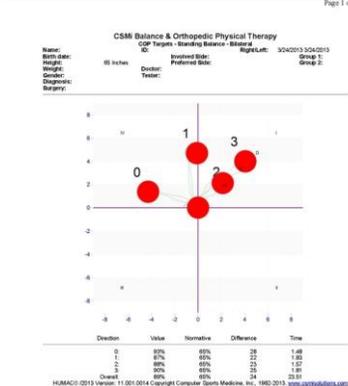Report	Description																														
 <table border="1"> <thead> <tr> <th>Direction</th> <th>Value</th> <th>Normative</th> <th>Difference</th> <th>Time</th> </tr> </thead> <tbody> <tr> <td>0</td> <td>87%</td> <td>85%</td> <td>22</td> <td>1:48</td> </tr> <tr> <td>1</td> <td>87%</td> <td>85%</td> <td>22</td> <td>1:48</td> </tr> <tr> <td>2</td> <td>88%</td> <td>85%</td> <td>23</td> <td>1:48</td> </tr> <tr> <td>3</td> <td>85%</td> <td>85%</td> <td>24</td> <td>2:03</td> </tr> <tr> <td>Overall</td> <td>88%</td> <td>85%</td> <td>24</td> <td>20:03</td> </tr> </tbody> </table>	Direction	Value	Normative	Difference	Time	0	87%	85%	22	1:48	1	87%	85%	22	1:48	2	88%	85%	23	1:48	3	85%	85%	24	2:03	Overall	88%	85%	24	20:03	<ul style="list-style-type: none"> A plot of the targets and a tracing of the patient's COP during the test. The targets are numbered in the order they were selected during the test. The Score (percent of time) they were on-target. The Normative value (set with the File, Preferences command). The Difference between the Patient's Score and the Normative value. The Time (minutes:seconds) to reach each target.
Direction	Value	Normative	Difference	Time																											
0	87%	85%	22	1:48																											
1	87%	85%	22	1:48																											
2	88%	85%	23	1:48																											
3	85%	85%	24	2:03																											
Overall	88%	85%	24	20:03																											

Options

Termination

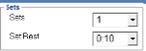Display	Parameter	Definition
	Sets	Number of sets to perform
	Set Rest	Rest period (seconds) at the end of each set

Settings

Display	Parameter	Definition
	Hold Time	Time in seconds the patient should remain on the target. Note: The FallTrak II counts-down the Hold Time so the patient knows when they should move to the next target.


Range of Motion

Display	Parameter	Definition
	Set AZ	Set the patient's neutral standing position. Ask the patient to stand so they feel balanced Right/Left and Anterior/Posterior
	Start/Stop	Measure the patient's maximum R/L and A/P range. <ol style="list-style-type: none">1. Have the patient stand in a neutral position.2. Click the start button.3. Have the patient lean R/L and A/P.4. Click the Stop button.

Random Motion (Force Plate Mode or Tilt Mode)

Provide a randomly moving target for the patient to follow.

Display

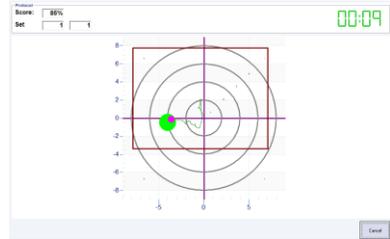Display	Description
	<ul style="list-style-type: none"> • Provide a randomly moving target for the patient to follow. • Their score is the percent of time they were on the target. • If a Time is specified, the Timer counts-down, otherwise it counts up. • A red box indicates the patient's Range of Motion. The target moves randomly within this space.

Report

Report	Description
	<ul style="list-style-type: none"> • Display a tracing of the patient's movement during the exercise. • The Score is the percent of time they were on-target.

Options

Termination

Display	Parameter	Definition
	Sets	Number of sets to perform
	Set Rest	Rest period (seconds) at the end of each set
	Time	The duration (minutes:seconds) of each set.

Settings

Display	Parameter	Definition
	Speed	The speed the target moves around the screen.

Range of Motion



Display	Parameter	Definition
	Set AZ	Set the patient's neutral standing position. Ask the patient to stand so they feel balanced Right/Left and Anterior/Posterior
	Start/Stop	Measure the patient's maximum R/L and A/P range. <ol style="list-style-type: none"> 5. Have the patient stand in a neutral position. 6. Click the start button. 7. Have the patient lean R/L and A/P. 8. Click the Stop button.

Force vs. Time (Force Plate Mode)

Force vs. Time plot used to measure a patient's single leg isometric weight bearing capabilities. Test Results display curves of the patient's force vs. time.

Display

Display	Description
	<ul style="list-style-type: none"> • The real-time force vs. time plots. • A cursor (dot) showing the patient's current force (Pounds or KG). • The cursor is green if the patient is within the Display Target range and red if they are outside the range. • The Peak Force during the test. • The current Rep and Number of Reps. • The current Set and Number of Sets. • A count-up timer showing the total test time.

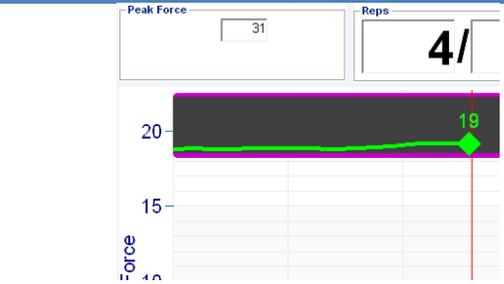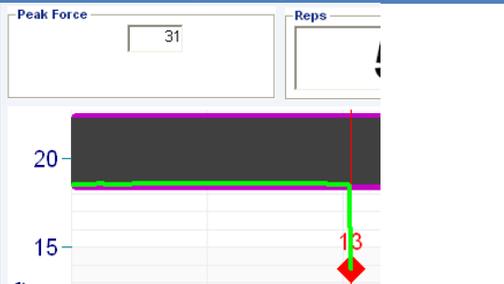In Target Range	Out Of Target Range
 <p>Figure 4 Force in Target</p>	 <p>Figure 5 Force Out of Target</p>

Report


Report	Description
	<ul style="list-style-type: none"> • Display the patient's Force vs. Time plots. • The Peak Force, Average Force, Peak Force Slope, Time to Half Peak Force and Time to Peak Force.

Options

Termination

Display	Parameter	Definition
	Sets	Number of sets to perform
	Set Rest	Rest period (seconds) at the end of each set
	Reps	The number of repetitions to perform.

Settings

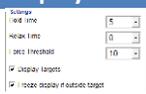Display	Parameter	Definition
	Hold Time	Time (seconds) the patient should exert force on the platform for each repetition
	Relax Time	Time (seconds) at the end of each repetition
	Force Threshold	Minimum force the patient must meet for the Hold Time to begin
	Display Targets	Display minimum/maximum force targets for the patient to achieve. Note: You can move the force targets by clicking and dragging them with the mouse.
	Freeze Display	Stop the Hold Time counter if the patient is not within the Targets .

Force Limits

Note: If **Display Targets** is not selected, **Force Limits** are not used.

Display	Parameter	Definition
	Delete	Clear the force limits.
	Scale	The red-marker indicates the current force on the balance platform. The green band indicates the upper and lower force limits. To have the patient load one leg at 50 to 60% of their body weight, set the Lower to 50% and the Upper to 60% and have the patient step on to the platform to measure their body weight.
	Upper %	The percent of the maximum force used for the Upper Display Target. For example if you were exercising the patient involved side and wanted them to limit their force to 50% of their body weight you would have them stand on the platform to record the weight, set the Upper% to 50% and enable Display Targets and Freeze Display if Outside Target .
	Lower%	The percent of the maximum force used for the Lower Display Target. For example if you were exercising the patient involved side and wanted them to achieve a minimum of 25% of their body weight you would have them stand on the platform to record the weight, set the Lower% to 25% and enable Display Targets and Freeze Display if Outside Target .

Scale (Force Plate Mode)

Show the patient's force on the platform using a scale display.

Display

Display	Description
	<ul style="list-style-type: none"> • A real-time pointer of the Force. • A lazy arm showing the Peak Force for the current repetition. • A Yellow band indicating the Forces below the Lower Target. • A Green band indicating Forces in the Target range. • A Red band indicating the Forces below the Lower Target. • The pointer color indicates if the force is Below, In, or Above the Target Range. • The current Rep and Number of Reps. • The current Set and Number of Sets. • A count-up timer showing the total test time.

Report


Report	Description																								
<table border="1"> <caption>Performance Metrics</caption> <thead> <tr> <th>Parameter</th> <th>Value</th> <th>Goal Val</th> <th>Unit</th> </tr> </thead> <tbody> <tr> <td>Peak Force (Pounds)</td> <td>30</td> <td>0.02</td> <td>12</td> </tr> <tr> <td>Average Force (Pounds)</td> <td>18</td> <td>0.03</td> <td>11</td> </tr> <tr> <td>Peak Force Slope (Pounds/Second)</td> <td>13</td> <td>0.42</td> <td>8</td> </tr> <tr> <td>Time to Half Peak Force (Seconds)</td> <td>0.00</td> <td>0.00</td> <td></td> </tr> <tr> <td>Time to Peak Force (Seconds)</td> <td>3.18</td> <td>0.40</td> <td></td> </tr> </tbody> </table>	Parameter	Value	Goal Val	Unit	Peak Force (Pounds)	30	0.02	12	Average Force (Pounds)	18	0.03	11	Peak Force Slope (Pounds/Second)	13	0.42	8	Time to Half Peak Force (Seconds)	0.00	0.00		Time to Peak Force (Seconds)	3.18	0.40		<ul style="list-style-type: none"> • Display the patient's Force vs. Time plots. • The Peak Force, Average Force, Peak Force Slope, Time to Half Peak Force and Time to Peak Force.
Parameter	Value	Goal Val	Unit																						
Peak Force (Pounds)	30	0.02	12																						
Average Force (Pounds)	18	0.03	11																						
Peak Force Slope (Pounds/Second)	13	0.42	8																						
Time to Half Peak Force (Seconds)	0.00	0.00																							
Time to Peak Force (Seconds)	3.18	0.40																							

Options

Termination


Display	Parameter	Definition
	Sets	Number of sets to perform
	Set Rest	Rest period (seconds) at the end of each set
	Reps	The number of repetitions to perform.

Settings

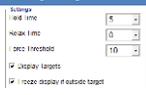Display	Parameter	Definition
	Hold Time	Time (seconds) the patient should exert force on the platform for each repetition
	Relax Time	Time (seconds) at the end of each repetition
	Force Threshold	Minimum force the patient must meet for the Hold Time to begin
	Display Targets	Display minimum/maximum force targets for the patient to achieve. Note: You can move the force targets by clicking and dragging them with the mouse.
	Freeze Display	Stop the Hold Time counter if the patient is not within the Targets .

Force Limits

Note: If **Display Targets** is not selected, **Force Limits** are not used.

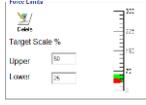Display	Parameter	Definition
	Delete	Clear the force limits.
	Scale	The red-marker indicates the current force on the balance platform. The green band indicates the upper and lower force limits. To have the patient load one leg at 50 to 60% of their body weight, set the Lower to 50% and the Upper to 60% and have the patient step on to the platform to measure their body weight.
	Upper %	The percent of the maximum force used for the Upper Display Target. For example if you were exercising the patient involved side and wanted them to limit their force to 50% of their body weight you would have them stand on the platform to record the weight, set the Upper% to 50% and enable Display Targets and Freeze Display if Outside Target .
	Lower%	The percent of the maximum force used for the Lower Display Target. For example if you were exercising the patient involved side and wanted them to achieve a minimum of 25% of their body weight you would have them stand on the platform to record the weight, set the Lower% to 25% and enable Display Targets and Freeze Display if Outside Target .

Tilt (Force Plate Mode or Tilt Mode)

Shows the platform tilt in the Right/Left and Anterior/Posterior planes.

Display

Display	Description
	<ul style="list-style-type: none"> • Real-time pointers of the Tilt. • Lazy arms showing the maximum tilt in each direction. • A count-up timer showing the total test time.

Options


Termination

Display	Parameter	Definition
	Sets	Number of sets to perform
	Set Rest	Rest period (seconds) at the end of each set
	Time	The duration (minutes:seconds) of each set.


Roadway (Force Plate Mode or Tilt Mode)

Roadway can be used to measure a patient's ability to weight shift ((Force Plate Mode or Tilt Mode)) or load the involved limb ((Force Plate Mode). Test results report the accuracy of the patient's ability to weight shift or load their limb.

Display

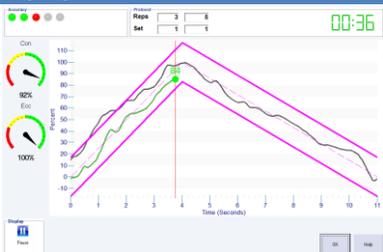Display	Description
	<ul style="list-style-type: none"> The goal is for the patient to keep the round cursor between the roadway boundaries. (on-target). In COP mode, the ball position corresponds to the patient's COP in the R/L or A/P plane. The Y-Axis displays the percent of the patient's total ROM. In Force mode, the ball position corresponds to the patient's force production. The Y-Axis displays the percent of the patient's target force. A green ball color indicates the patient is on-target, a red ball color indicates they are not on-target. The Score dials indicate the percent of time the patient was on-target during the Concentric and Eccentric phases of the exercise. The display shows the target in Magenta, the current repetition in yellow and the previous 9 repetitions in gray. Pause/Continue buttons to freeze, un-freeze the display. This allows you to instruct the patient if they have questions on the display. The Accuracy area shows the number of repetitions where the patient was within (green light) or below (red light) the target accuracy set in the Protocol for both the Concentric and Eccentric contraction. If the patient has three repetitions where either the Concentric or Eccentric score is below the Accuracy setting the display is halted.

Report

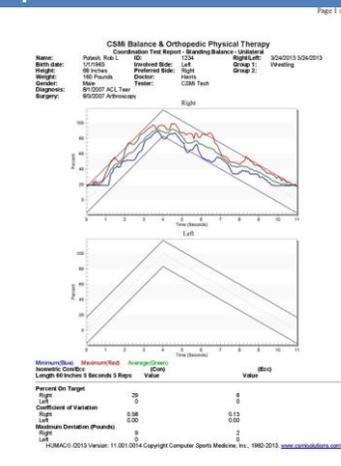Report	Description
	<ul style="list-style-type: none"> Display the patient's COP or Position vs. Time overlaid on the roadway. The Peak Force, Average Force, Peak Force Slope, Time to Half Peak Force and Time to Peak Force.

Options

Termination

Display	Parameter	Definition
	Sets	Number of sets to perform
	Set Rest	Rest period (seconds) at the end of each set
	Reps	The number of repetitions to perform.

Mode and Action

Display	Parameter	Definition
	COP	Display feedback based on the patient's Center of Pressure.
	>	Exercise the Patient in the Right/Left plane.
	^	Exercise the Patient in the Anterior/Posterior plane.
	Force	Display feedback based on the patient's Force on the platform.

Settings


Display	Parameter	Definition
	Profile	The Roadway Pattern. Note: You can customize the patterns with the Utilities, Interactive Feedback Editor command.
	Level	The width of the Roadway.
	Accuracy	The percent on-target the Con and Ecc phases of a repetition must be to count as a "good rep". Note: Set this value to 0 if you do not want the FallTrak II to track accuracy.

Range of Motion

Display	Parameter	Definition
	Set AZ	Set the patient's neutral standing position. Ask the patient to stand so they feel balanced Right/Left and Anterior/Posterior
	Start/Stop	Measure the patient's maximum R/L and A/P range. <ol style="list-style-type: none"> 1. Have the patient stand in a neutral position. 2. Click the start button. 3. Have the patient lean R/L and A/P. 4. Click the Stop button.

Force Limits


Note: If **Display Targets** is not selected, **Force Limits** are not used.

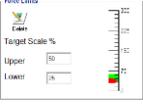Display	Parameter	Definition
	Delete	Clear the force limits.
	Scale	The red-marker indicates the current force on the balance platform. The green band indicates the upper and lower force limits. For example, to have the patient load one leg at 50 to 60% of their body weight, set the Lower to 50% and the Upper to 60% and the patient step on to the platform to measure their body weight.
	Upper %	The percent of the maximum force used for the Upper Display Target. For example if you were exercising the patient involved side and wanted them to limit their force to 50% of their body weight you would have them stand on the platform to record the weight, set the Upper% to 50% and enable Display Targets and Freeze Display if Outside Target .
	Lower%	The percent of the maximum force used for the Lower Display Target. For example if you were exercising the patient involved side and wanted them to achieve a minimum of 25% of their body weight you would have them stand on the platform to record the weight, set the Lower% to 25% and enable Display Targets and Freeze Display if Outside Target .

Pong (Force Plate Mode or Tilt Mode)

Pong has the patient play a tennis-like game against the computer.

Display

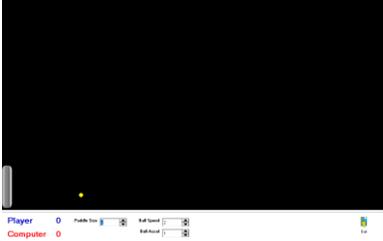Display	Description
	<ul style="list-style-type: none"> The tennis-like display with Player and Computer paddles. The Player and Computer score. Controls to adjust the Paddle Size, Ball Speed and Ball Acceleration.

Report

Reports are not available for the games.

Options

Termination

Display	Parameter	Definition
	Sets	Number of sets to perform
	Set Rest	Rest period (seconds) at the end of each set
	Nbr Balls	The number of balls to play.

Mode and Action

Display	Parameter	Definition
	COP	Display feedback based on the patient's Center of Pressure.
	>	Exercise the Patient in the Right/Left plane.
	^	Exercise the Patient in the Anterior/Posterior plane.
	Force	Display feedback based on the patient's Force on the platform. (Force Plate Mode)

Settings

Display	Parameter	Definition
	Ball Speed	Ball Speed from 1 (Slowest) to 25 (Fastest).
	Ball Accel	Ball Acceleration (change in speed) after each paddle strike from 0 (No Acceleration) to 10 (Maximum Acceleration).
	Ball Size	Ball Size from 1 (Smallest) to 3 (Biggest).
	Ball Rest	Rest time (seconds) before each Ball launch.
	Computer Skill	Computer Skill playing against the patient from 0 (Beginner) to 100 (Expert).
	Paddle Size	The patient's paddle size from 1 (Smallest) to 10 (Biggest).

Range of Motion

Display	Parameter	Definition
	Set AZ	Set the patient's neutral standing position. Ask the patient to stand so they feel balanced Right/Left and Anterior/Posterior
	Start/Stop	<p>Measure the patient's maximum R/L and A/P range.</p> <ol style="list-style-type: none"> 5. Have the patient stand in a neutral position. 6. Click the start button. 7. Have the patient lean R/L and A/P. 8. Click the Stop button.

Force Limits (Force Plate Mode)


Note: If **Display Targets** is not selected, Force Limits are not used.

Display	Parameter	Definition
	Delete	Clear the force limits.
	Scale	The red-marker Scale indicates the current force on the balance platform. The green band indicates the upper and lower force limits. For example, to have the patient load one leg at 50 to 60% of their body weight, set the Lower to 50% and the Upper to 60% and the patient step on to the platform to measure their body weight.
	Upper %	The percent of the maximum force used for the Upper Display Target. For example if you were exercising the patient involved side and wanted them to limit their force to 50% of their body weight you would have them stand on the platform to record the weight, set the Upper% to 50% and enable Display Targets and Freeze Display if Outside Target .
	Lower%	The percent of the maximum force used for the Lower Display Target. For example if you were exercising the patient involved side and wanted them to achieve a minimum of 25% of their body weight you would have them stand on the platform to record the weight, set the Lower% to 25% and enable Display Targets and Freeze Display if Outside Target .

Breakout (Force Plate Mode or Tilt Mode)

Breakout has the patient move a paddle to knock blocks out of a wall.

Display

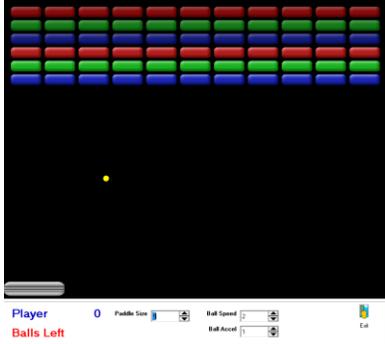Display	Description
	<ul style="list-style-type: none"> • The breakout display with the Player paddle. • The Player Score (blocks destroyed). • The number of Balls left. • Controls to adjust the Paddle Size, Ball Speed and Ball Acceleration.

Report

Reports are not available for the games.

Options


Termination

Display	Parameter	Definition
	Sets	Number of sets to perform
	Set Rest	Rest period (seconds) at the end of each set
	Nbr Balls	The number of balls to play.

Mode and Action

Display	Parameter	Definition
	COP	Display feedback based on the patient's Center of Pressure.
	>	Exercise the Patient in the Right/Left plane.
	^	Exercise the Patient in the Anterior/Posterior plane.
	Force	Display feedback based on the patient's Force on the platform. (Force Plate Mode)

Settings

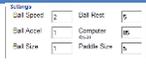Display	Parameter	Definition
	Ball Speed	Ball Speed from 1 (Slowest) to 25 (Fastest).
	Ball Accel	Ball Acceleration (change in speed) after each paddle strike from 0 (No Acceleration) to 10 (Maximum Acceleration).
	Ball Size	Ball Size from 1 (Smallest) to 3 (Biggest).
	Ball Rest	Rest time (seconds) before each Ball launch.
	Computer Skill	Computer Skill playing against the patient from 0 (Beginner) to 100 (Expert).
	Paddle Size	The patient's paddle size from 1 (Smallest) to 10 (Biggest).

Range of Motion

Display	Parameter	Definition
	Set AZ	Set the patient's neutral standing position. Ask the patient to stand so they feel balanced Right/Left and Anterior/Posterior
	Start/Stop	<p>Measure the patient's maximum R/L and A/P range.</p> <ol style="list-style-type: none"> 9. Have the patient stand in a neutral position. 10. Click the start button. 11. Have the patient lean R/L and A/P. 12. Click the Stop button.

Force Limits (Force Plate Mode)

Note: If **Display Targets** is not selected, Force Limits are not used.

Display	Parameter	Definition
	Delete	Clear the force limits.
	Scale	The red-marker Scale indicates the current force on the balance platform. The green band indicates the upper and lower force limits. For example, to have the patient load one leg at 50 to 60% of their body weight, set the Lower to 50% and the Upper to 60% and the patient step on to the platform to measure their body weight.
	Upper %	The percent of the maximum force used for the Upper Display Target. For example if you were exercising the patient involved side and wanted them to limit their force to 50% of their body weight you would have them stand on the platform to record the weight, set the Upper% to 50% and enable Display Targets and Freeze Display if Outside Target .
	Lower%	The percent of the maximum force used for the Lower Display Target. For example if you were exercising the patient involved side and wanted them to achieve a minimum of 25% of their body weight you would have them stand on the platform to record the weight, set the Lower% to 25% and enable Display Targets and Freeze Display if Outside Target .

Balance Board (Force Plate Mode or Tilt Mode)

Two-dimensional balance game where player must tilt the board (by leaning R/L, A/P) to move the ball through a maze and into the target hole. 12 levels of increasing difficulty are available.

Display

Display	Description
	<ul style="list-style-type: none"> • The Balance Board Display. • A count-up timer of the total time played. • The current level. • A Menu button.

Report

Reports are not available for the games.

Options

Menu

Display	Description
	<ul style="list-style-type: none"> • Level: Select the Level to play and begin the game. • Play: Display the Levels. • Player: Select the player. • Scores: View the current player's scores. • Help: Display Help System. • Quit: Return to the Dashboard.

Range of Motion

Display	Parameter	Definition
	Set AZ	Set the patient's neutral standing position. Ask the patient to stand so they feel balanced Right/Left and Anterior/Posterior
	Start/Stop	Measure the patient's maximum R/L and A/P range. <ol style="list-style-type: none"> 1. Have the patient stand in a neutral position. 2. Click the start button. 3. Have the patient lean R/L and A/P. 4. Click the Stop button.

Skier (Force Plate Mode or Tilt Mode)

Two-dimensional ski-race game where patient must ski down a course. The patient's Right/Left tilt controls the skier's Right/Left position. The patient's Anterior/Posterior tilt controls the skier's speed. 12 levels of increasing difficulty are available.

Display

Display	Description
	<ul style="list-style-type: none"> • The Skier Display. • The patient leans Right/Left to steer the skier. • The patient leans Anterior/Posterior to increase/decrease the skier's speed. • A count-up timer of the total time played. • The current level. • A Menu button.

Report

Reports are not available for the games.

Options

Menu

Display	Description
	<ul style="list-style-type: none"> • Level: Select the Level to play and begin the game. • Play: Display the Levels. • Player: Select the player. • Scores: View the current player's scores. • Help: Display Help System. • Quit: Return to the Dashboard.

Range of Motion

Display	Parameter	Definition
	Set AZ	Set the patient's neutral standing position. Ask the patient to stand so they feel balanced Right/Left and Anterior/Posterior
	Start/Stop	<p>Measure the patient's maximum R/L and A/P range.</p> <ol style="list-style-type: none"> 1. Have the patient stand in a neutral position. 2. Click the start button. 3. Have the patient lean R/L and A/P. 4. Click the Stop button.

Snowboarder (Force Plate Mode or Tilt Mode)

Two-dimensional snowboard-race game where patient must ski down a course. The patient's Anterior/Posterior tilt controls the snowboarder's Right/Left position. The patient's Right/Left tilt controls the skier's speed. 12 levels of increasing difficulty are available.

Note: The balance platform should be oriented so the player's front foot is toward the display and the USB cable exits the board in front of the player.

Display

Display	Description
	<ul style="list-style-type: none"> • The Snowboarder Display. • The patient leans Right/Left to steer the snowboarder. • The patient leans Anterior/Posterior to increase/decrease the snowboarder's speed. • A count-up timer of the total time played. • The current level. • A Menu button.

Report

Reports are not available for the games.

Options

Menu

Display	Description
	<ul style="list-style-type: none"> • Level: Select the Level to play and begin the game. • Play: Display the Levels. • Player: Select the player. • Scores: View the current player's scores. • Help: Display Help System. • Quit: Return to the Dashboard.

Range of Motion

Display	Parameter	Definition
	<p>Set AZ</p> <hr/> <p>Start/Stop</p>	<p>Set the patient's neutral standing position. Ask the patient to stand so they feel balanced Right/Left and Anterior/Posterior</p> <hr/> <p>Measure the patient's maximum R/L and A/P range.</p> <ol style="list-style-type: none"> 1. Have the patient stand in a neutral position. 2. Click the start button. 3. Have the patient lean R/L and A/P. 4. Click the Stop button.

Mode and Action

Display	Parameter	Definition
	<p>Mode</p>	<p>Select if the Snowboarder is Regular (left foot forward) or Goofy (right foot forward).</p>

Pilot (Force Plate Mode or Tilt Mode)

Three-dimensional flight-race game where patient must fly through targets. The patient's Right/Left tilt controls the plane's Right/Left position. The patient's Anterior/Posterior tilt controls the plane's Climb/Descend speed. 12 levels of increasing difficulty are available.

Display

Display	Description
	<ul style="list-style-type: none"> • The Pilot Display. • The patient leans Right/Left to turn the plane Right/Left. • The patient leans Anterior/Posterior to have the plane Climb/Descend. • A count-up timer of the total time played. • The current level. • A Menu button.

Report

Reports are not available for the games.

Options

Menu

Display	Description
	<ul style="list-style-type: none"> • Level: Select the Level to play and begin the game. • Play: Display the Levels. • Player: Select the player. • Scores: View the current player's scores. • Help: Display Help System. • Quit: Return to the Dashboard.

Range of Motion

Display	Parameter	Definition
	<p>Set AZ</p> <hr/> <p>Start/Stop</p>	<p>Set the patient's neutral standing position. Ask the patient to stand so they feel balanced Right/Left and Anterior/Posterior</p> <hr/> <p>Measure the patient's maximum R/L and A/P range.</p> <ol style="list-style-type: none"> 1. Have the patient stand in a neutral position. 2. Click the start button. 3. Have the patient lean R/L and A/P. 4. Click the Stop button.

The Main FallTrak II Screen

The Main FallTrak II Screen contains a number of components:

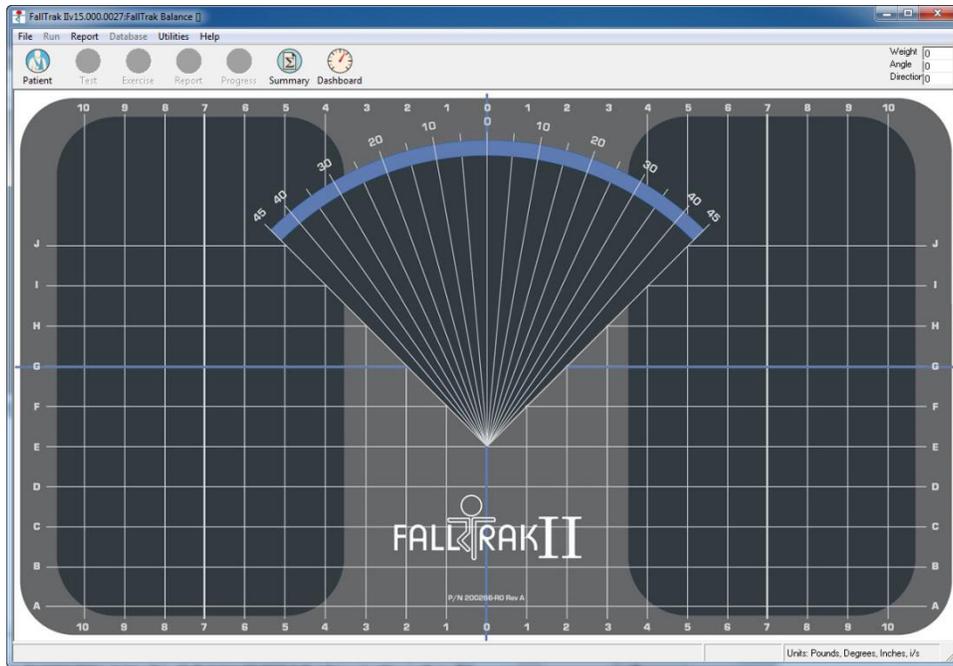


Figure 6 FallTrak II Form

The **Title Bar** located at the top of the screen contains the Application Name and Version (FallTrak II Version 11.001.0016), the machine (Balance) and the current patient (Potash, Robert).

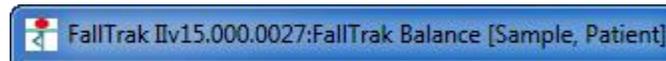


Figure 7 Title Bar

Just below the title bar is the **Menu Bar**. You can access all of the FallTrak II functions from the menu bar. Each menu item has an underlined character. Holding the **Alt** key while pressing the underlined character selects the menu item. For example **Alt-F** selects the file menu.

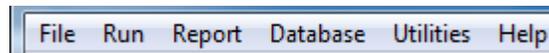


Figure 8 Menu Bar

The **Button Bar** contains shortcuts to the most commonly used FallTrak II functions. When the FallTrak II program begins, only the **Patient**, **Dashboard**, and **Help** options are available. Selecting a patient enables the **Test**, **Exercise**, and **Report** options.



Figure 9 Button Bar No Patient Selected



Figure 10 Button Bar Patient Selected

To the right of the button bar is a real-time display of the Balance data: **Weight**, **Tilt Angle**, and **Tilt Direction**.

Weight	-0.02
Angle	0
Direction	0

Figure 11 Real-Time Display

The bottom of the screen contains the **Status Bar** which displays the current test pattern and system units. The system units are displayed for the Force, Position, and Velocity measures.

Balance: N/A	Units: Pounds, Inches, i/s
--------------	----------------------------

Figure 12 Status Bar

Starting FallTrak II Program

Steps

1. From the Menu Bar **File** option, select **Preferences**.

Figure 13 Select Machine Type

2. From the **Interface** area select **USB/COMM (Balance)**.



Figure 14 Select USB Port

3. Click the **OK** button to save your changes. The FallTrak II will default to these settings the next time the program is started.
4. If the FallTrak II Balance drivers are installed and the FallTrak II Balance Board is plugged into the USB Port, the following dialog box is displayed.

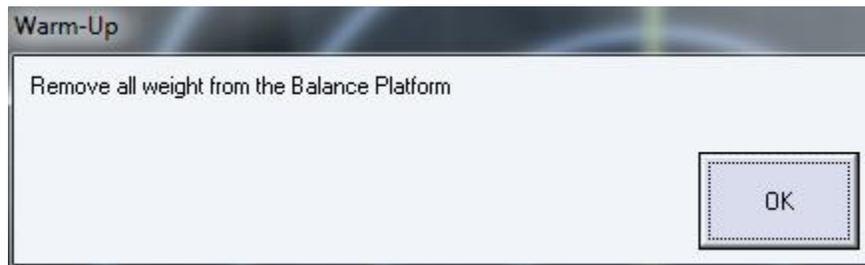


Figure 15 Warm-up Screen

5. Insure the platform is sitting on a flat surface and remove all weight from the Platform (no one is standing on the Platform).
6. Click the **OK** button to read the zero weight. The FallTrak II will return to the Main Menu.

Testing a Patient

The process to test a patient is:

1. Select a Patient.
2. Select a Test.
3. Select a Protocol.
4. Run the Test.
5. Print a Report.

The following example demonstrates running and printing the results of a CTSIB test.

Note:

- When using a Balance System any barrier (shoes, socks) between the patient and the FallTrak II Balance Board surface may affect their performance. In some situations you may want to remove all barriers. It depends on if you want to measure absolute vs. real-world performance. The normative data used in the FallTrak II Balance CTSIB Report assumes no shoes with socks.
- Whichever condition testing you use is your decision. However if you plan to do follow-up comparisons it is important to keep all variables the same from test to test.

Patient Selection

The Select a Patient screen allows you to add, edit, or delete patients from the database.

Steps

1. From the Main Screen, click the **Patient** button.
2. The FallTrak II will display the **Patient Selection Screen**.
3. For this example we will add a new patient. Click the **New** button.

Screen

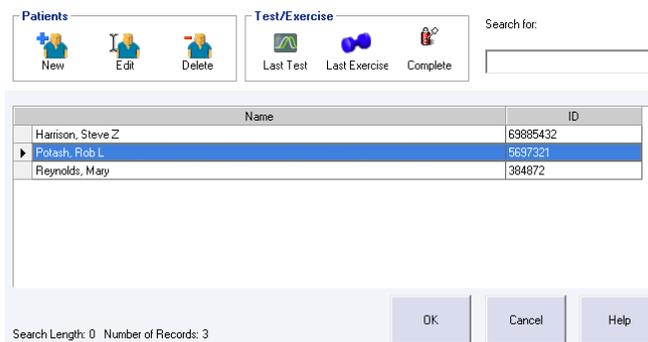


Figure 16 Patient Selection

Actions

Action	Definition
OK	Select the highlighted patient. You can also double-click a patient's name to select them.
Cancel	Cancel the patient selection and return to the main FallTrak II screen.
Search	As you begin typing the patient's last name, the FallTrak II automatically displays patients matching the search criteria.
New	Add a new patient to the database.
Edit	Edit the highlighted patient's background information (Name, Height, Weight, etc.).
Delete	Delete the highlighted patient from the FallTrak II Database. IMPORTANT: <i>When you delete a patient from the FallTrak II Database, all of their test and exercise data is deleted from the system. There is no way to recover deleted data.</i>
Last Test	Repeat the last test run on the patient. The FallTrak II recalls the most recent test run on the patient and begins a new test session using the same pattern and protocol.
Last Exercise	Repeat the last exercise run on the patient. The FallTrak II recalls the most recent exercise run on the patient and begins a new exercise session using the same pattern and protocol.
Complete	Complete the last test or exercise run on the patient. The FallTrak II recalls the patient's previous test and goes directly to the test status screen. This is typically used if you are unable to complete the test, e.g. the patient stopped or there was a computer problem.

Adding a New Patient

Steps

1. The FallTrak II will display a blank patient information screen. The patient information entered here will appear at the top of every report for the patient and can be used as filters in the Group Summary/Normative Database module. We recommend you complete as many fields as possible when adding a new patient.
2. After adding the patient information click **OK** to save new Patient information and return to the main FallTrak II Screen. The patient name will then appear in the Title Bar and all of the buttons on the Button Bar including Test, Exercise, and Report will be active.

Screen

The screenshot shows the 'Patient Background Information' dialog box with the 'General' tab selected. The 'Injury/Group' tab is also visible and contains the text 'INRTEK'. The 'General' tab contains the following fields: 'Last First MI' with text boxes for 'Potash', 'Rob', and a dropdown for 'L'; 'ID' with a text box for '1234'; 'Birth date' with a date picker for '1/1/1963'; 'Height' with a text box for '172.72' and a unit dropdown for 'Centimeters'; 'Weight' with a text box for '72.8' and a unit dropdown for 'Kilograms'; 'Gender' with radio buttons for 'Male' (selected) and 'Female'; 'Preferred Side' with checkboxes for 'Left' and 'Right' (selected); 'Involved Side' with checkboxes for 'Left' (selected) and 'Right'; and 'Tester' with a dropdown menu showing 'Potash, Rich'. At the bottom are 'OK', 'Cancel', and 'Help' buttons.

Figure 17 Patient Information - General


The screenshot shows the 'Patient Background Information' dialog box with the 'Injury/Group' tab selected. The 'General' tab is also visible and contains the text 'INRTEK'. The 'Injury/Group' tab contains the following fields: 'Diagnosis' with a dropdown menu showing 'ACL Tear'; 'Injury Date' with a date picker for '8/1/2007'; 'Surgery' with a dropdown menu showing 'Arthroscopy'; 'Surgery Date' with a date picker for '8/3/2007'; 'Doctor' with a dropdown menu showing 'Harris'; 'Group 1' with a dropdown menu showing 'Wrestling'; and 'Group 2' with an empty dropdown menu. At the bottom are 'OK', 'Cancel', and 'Help' buttons.

Figure 18 Patient Information – Injury/Group

Fields

Field Name	Description
Last, First, Middle	Enter Patient name. Press Tab to advance to next field
ID	Patient ID number.
Height	Enter according to the displayed units of measure. <i>Required field.</i>
Weight	Enter according to the displayed units of measure. <i>Required field.</i>
Gender	<i>Required field.</i>
Preferred Side	Patient's dominant side.
Involved Side	Injured side.
Tester	Name of perform performing the test. The pull-down list attached to this field should be edited to include the names of the clinicians using the FallTrak II Balance.
Diagnosis	The pull-down list attached to this field should be edited to include the common diagnoses seen.
Injury Date	Pop-up calendar available.
Surgery	This pull-down list attached to this field should be edited to include the common surgeries seen
Doctor	The pull-down list attached to this field should be edited to include the names of the referring physicians
Group 1	Extra field to further define the patient. The pull-down list attached to this field can be edited.
Group 2	Second field to further define the patient. The pull-down list attached to this field can be edited.

Note:

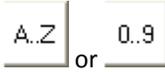

- Click the Keypad buttons  or  to pop-up a keypad for data entry with a touch-screen monitor.
- *After a patient has been tested, use the **Edit Background** command to edit the information stored with the test.*
- *The **Patient Selection** screen lists the Patient Name and ID. By default the list is sorted by patient name. To sort by ID, click on the ID column heading.*
- Only one copy of the patient's Name, ID, Birth Date, and Sex are stored in the FallTrak II database. Any changes to these fields will automatically appear on every test and exercise record for the patient.
- The Height, Weight, Preferred Side, Involved Side, Diagnosis, Injury Date, Surgery, Surgery Date, Doctor, Tester, Group1, and Group2 values are stored individually with each test and exercise session run. This allows changes to these fields to accurately reflect the patient's condition at the time of the test or exercise session. Use the **Edit Background** (Section 0) command to change the values for a test which has already been run.

Pattern Selection

Steps

1. From the FallTrak II Button Bar, click **Test**.
2. The FallTrak II will display the **Pattern Selection** screen.
3. The FallTrak II allows tests to be run in a bi-laterally (standing on both feet) or a uni-laterally (standing on one foot).
4. Click on **Standing Balance Bilateral** row to highlight it. *Note: Most Balance Tests are performed Bi-laterally.*
5. Click **OK** advance to the Protocol Selection Screen

Screen

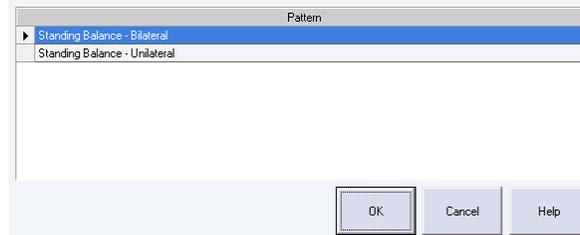


Figure 19 Pattern Selection

Actions

Field Name	Description
OK	Select the highlighted pattern. You can also double-click a pattern to select it.
Cancel	Cancel the pattern selection and return to the main FallTrak II screen.

Protocol Selection

Protocols are collections of one or more sets which describe the Display, Termination, Rest Period and other test parameters. A protocol can contain up to 25 sets.

Steps

1. In the **Feedback** area, select **CTSIB**.
2. The FallTrak II will display **Termination, Mode and Action** and **Settings** options applicable to the CTSIB test. **Note:** *The options for each feedback mode are described in the section titled **Error! Reference source not found.***
3. The FallTrak II will load the Protocol area, Select pull-down with all previous CTSIB protocols for this patient and all system-wide CTSIB protocols. For this example we will customize the protocol.
4. In the **Termination, Duration** area, set the **Time** to **0:30**.
5. In the **Settings** area, select **Eyes Open Firm Surface** and **Eyes Closed Firm Surface**.
6. In the **Protocol** area, **Rename** the protocol to **CTSIB – Firm – Eyes Open/Closed**. **Note:**
 - a. *The FallTrak II will remember this protocol the next time you test the patient.*
 - b. *Use to **Utilities, Protocol Editor** to create custom protocols which can be used with all patients.*
 - c. *Use the **Select** pull-down box to select a pre-defined custom protocol.*
7. Click the **OK** button to save the Protocol and move to the **Test Status** screen.

Screen

Figure 20 Protocol Selection

Actions

Field Name	Description
Select	The pull-down control displays a list of the protocols the patient has been tested with. The patient's most recent protocol at the top of the list. Following the patient's individual protocols, the FallTrak II displays the system protocols. System protocols are available to all patients and are created using Utilities, Protocol Editor from the Main Screen.
Rename	A text description which is displayed in the Protocol control after the protocol is saved from this screen.
OK	Continue with the displayed protocol.
Cancel	Cancel the test/exercise session and return to the main FallTrak II menu.
Termination	The number of sets and reps to be performed.
Mode and Action	The mode (Center of Pressure or Force) and the direction of the COP.
Settings	The settings for the feedback display.

Test Status

The Test Status screen displays all sets in the protocol and their status (Tested, Not Tested, Error).

Steps

1. Click the **All Sets** button to perform all of the sets in this protocol in sequential order.
2. The **Patient Setup** screen will appear.

Screen

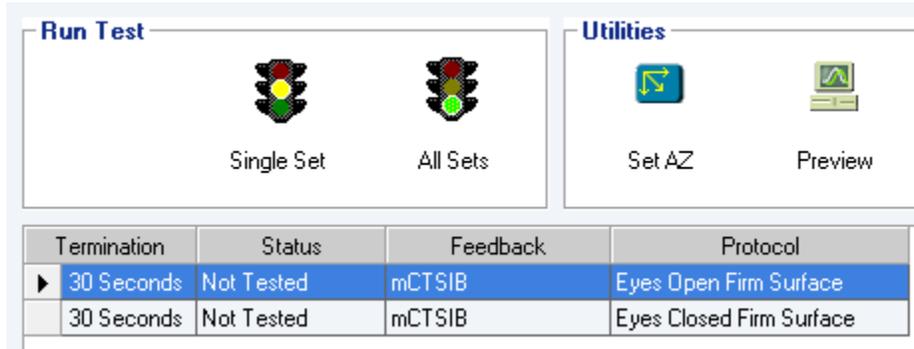


Figure 21 Test Status

Actions

Action	Definition
OK	Leave the Test Status screen and move to the Report screen. Once you leave the Test Status screen you cannot run additional sets from the protocol.
Side	Select the Side which will be tested. Note: <i>The side is not displayed for Bi-lateral patterns.</i>
Single Set	Run the currently highlighted set.
All Sets	Run all sets in the protocol starting with the selected set. Note: When you select All Sets the FallTrak II deletes all data from any sets which are run. For example, if you completed all sets in a 3 set protocol, select the second set and click All Sets, the FallTrak II will leave the data from the first set in-place and replace the data in the second and third set with the new data as the sets are re-run.
Set AZ	Re-set the patient's Foot Position. If you re-set the AZ you should re-run the sets.
Preview	Preview a test report. This allows you to quickly check the results of a set before leaving the Test Status screen.

Note: *If the patient stops in the middle of a set you may choose to re-run the set. When you re-run a set the FallTrak II overwrites the previous data with the new data.*

Status

Status	Definition
Not Tested	The set as not been run yet.
Tested	The set was completed successfully.
Error	An error occurred while running the set, for example, the patient did not go through a full range of motion or the clinician aborted the set.

Patient Setup

The Patient Setup screen allows you to record the patient's foot position on the platform. The foot position is used by the FallTrak II to compute the test results. If this setting is correct and the patient is standing in an anatomically correct position the patient COP should be centered on the Test Display screen.

Steps

1. Use the scales on the FallTrak II Balance Board to record the patient's foot position.
 - a. The **Angle** is read by comparing the foot angle to the angle display on the FallTrak II board.
 - b. The **Medial Malleolus** is read by selecting the letter row on the FallTrak II board with is closest to the Medial Malleolus.
 - c. The **Heel** is read by selecting the number column on the FallTrak II board with is closest to the centerline of the heel.
2. Enter the **Angle**, **Medial Malleolus** and **Heel** positions.
3. Click the **OK** button to save the settings. *Note: Once these positions are recorded, the patient should not move their feet for the duration of the test.*
4. The FallTrak II will display the **Anatomical Zero** screen.

Screen

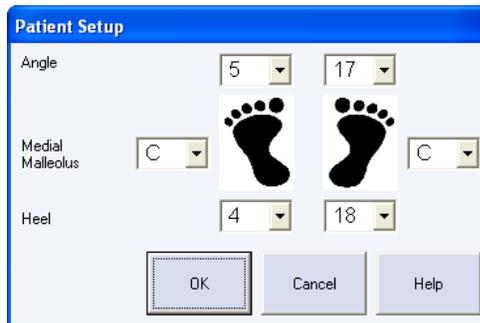


Figure 22 Patient Setup Screen

Anatomical Zero

If the patient setup values are correct and the patient is standing in a perfectly neutral position the patient COP should be centered on the Test Display screen. Many times due to body mechanics a patient is not centered even though their foot position is recorded. For this reason there is the option to have the FallTrak II Center the COP display.

Steps

1. Instruct the patient to stand in a neutral position.
2. Click the **OK** button to record this as their Anatomical Zero.
3. The FallTrak II will move to the Patient Feedback screen.

Screen

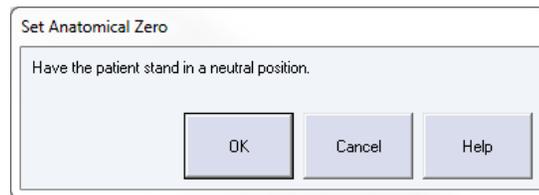


Figure 23 Set Anatomical Zero Screen

Move to COP Timer

Before tests which require the patient to start in the COP, the FallTrak II displays a 5 second “Move to Center of Pressure” timer. During this time the patient can see their COP on the display but no data is collected.

Steps

1. Instruct the patient to adjust their balance so the COP marker (magenta ball) is in the center of the screen.

Display



Figure 24 Move to COP Timer

Patient Feedback (CTSIB Test)

Steps

The CTSIB measures the patient's ability to hold their Center of Pressure which is where the crosshairs meet on the center of the screen.

1. Instruct the patient to remain as steady as possible.
2. After the timer counts down the FallTrak II will proceed to the next set. After all sets are completed the FallTrak II will return to the **Test Status** screen.

Screen

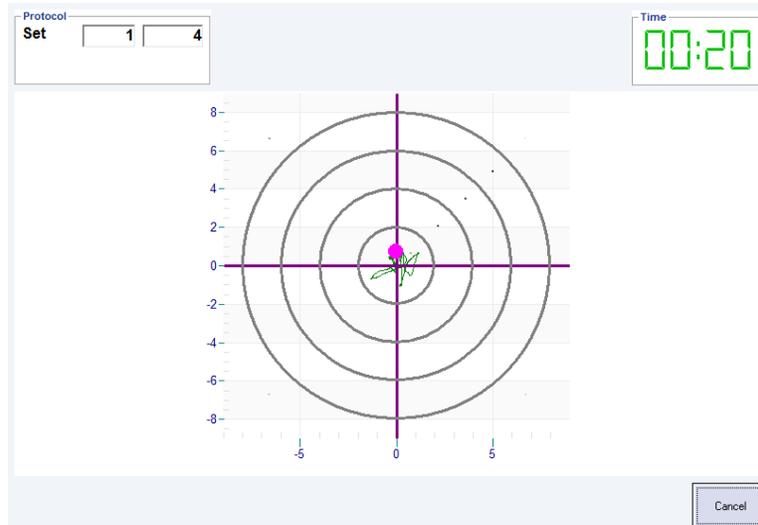


Figure 25 CTSIB Feedback Display (COP)

Test Status

At the conclusion of the test the FallTrak II will return to the Status screen. Notice all of the tests under Status now read **Tested**.

Steps

1. Click the **OK** button. The FallTrak II automatically stores the collected data in the database and proceeds to the **Report** screen.

Screen

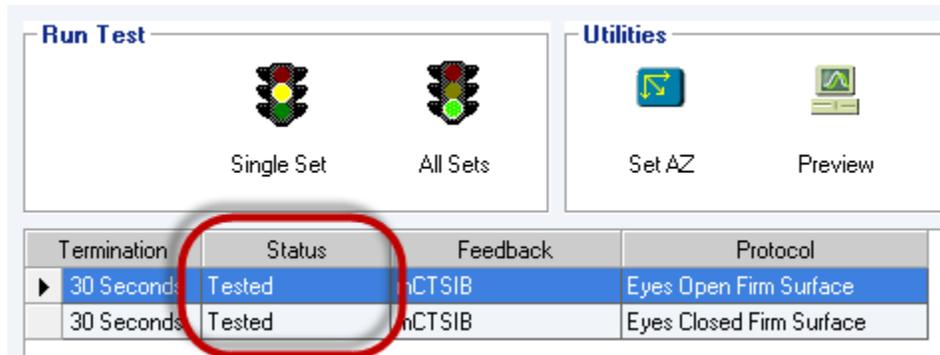


Figure 26 Status Screen - Tests Completed

REPORTS

Single Test Report

The FallTrak II report screen lists all of the stored test results and report options. Results are listed in chronological order with the most recent at the top. When you select a test from the listing grid, the FallTrak II automatically populates the reports types for the selected test.

Steps

1. From the Listing Grid, click the test to select it.
2. Click the Preview button to display the test report.

Screen

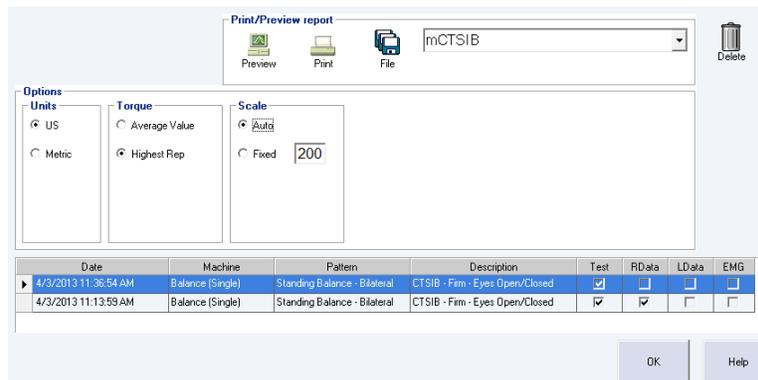


Figure 27 Reports Screen

Actions



Figure 28 Report: Actions

Status	Definition
Preview	Preview the selected report on the screen.
Print	Send the selected report to the printer.
File	Save the selected report to a disk file.
Type	Select the type of report to generate. When you select a report from the listing grid, the FallTrak II populates the Type list with all reports which are available for the selected test.
Delete	Delete the selected test from the database.

Options



Figure 29 Report: Options

Options

The **Options** section allows you to change how the data is processed and reported for the selected test.

Status	Definition
Units	Display results in US or Metric Units.
Torque	Report the Average Value or Highest Rep for Torque vs. Time and Scale tests.
Scale	Auto-scale or use a fixed scale for Torque vs. Time and Scale tests.

Note: Changes to the **options** only apply to the current report being generated and do not affect the **Preferences**. Use the **File, Preferences** command to set the default report options.

Progress Report

The FallTrak II Progress Report screen allows you to select two tests and print a Progress Report showing the initial values, the follow-up values and the change which occurred between the two.

Steps

1. From Reports menu, select Progress. The FallTrak II will display the Progress Report screen.
2. From the Listing Grid, select the Initial test and click the Initial button.
3. From the Listing Grid, select the Follow-up test and click the Follow-Up button.
4. Click the Preview button to display the report. The FallTrak II will confirm the protocols in the two tests match and if so, display the progress report.

Screen

Date	Machine	Pattern	Description	Test	PDdata	LDdata	EMG
4/3/2013 11:36:54 AM	Balance (Single)	Standing Balance - Bilateral	CTSIB - Firm - Eyes Open/Closed	<input checked="" type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>
4/3/2013 11:13:59 AM	Balance (Single)	Standing Balance - Bilateral	CTSIB - Firm - Eyes Open/Closed	<input checked="" type="checkbox"/>	<input checked="" type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>

Figure 30 Progress Report Screen

Involved Side Re-Test Report

The Involved Side Re-test Report allows you to combine the results of an initial test for one side with the results of a follow-up test for the other side into a single report. This is typically used when you test both side during an initial evaluation and only re-test the involved side during a follow-up visit.

Steps

1. From Reports menu, select **Involved Re-Test**. The FallTrak II will display the Re-Test Report screen.
2. From the Listing Grid, select the Right Side test and click the Right button.
3. From the Listing Grid, select the Left Side up test and click the Left button.
4. Click the Preview button to display the report. The FallTrak II will confirm the protocols in the two tests match and if so, display the Re-Test report.

Screen

Date	Machine	Pattern	Description	Test	PDdata	LDdata	EMG
4/3/2013 11:36:54 AM	Balance (Single)	Standing Balance - Bilateral	CTSIB - Firm - Eyes Open/Closed	<input checked="" type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>
4/3/2013 11:13:59 AM	Balance (Single)	Standing Balance - Bilateral	CTSIB - Firm - Eyes Open/Closed	<input checked="" type="checkbox"/>	<input checked="" type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>

Figure 31 Re-Test Report Screen

DASHBOARD

The FallTrak II Dashboard allows you to control the basic functions of the FallTrak II and provide patient feedback without running a test or exercise protocol. Reports can be printed but data is not saved when using the Dashboard and the Patient's information does on appear at the top of report. **Note:** *The Dashboard Options are explained in detail in "Feedback Displays".*

Steps

1. From the Button Bar click the **Dashboard** button or from the **Utilities** menu, select **Dashboard**.

Screen

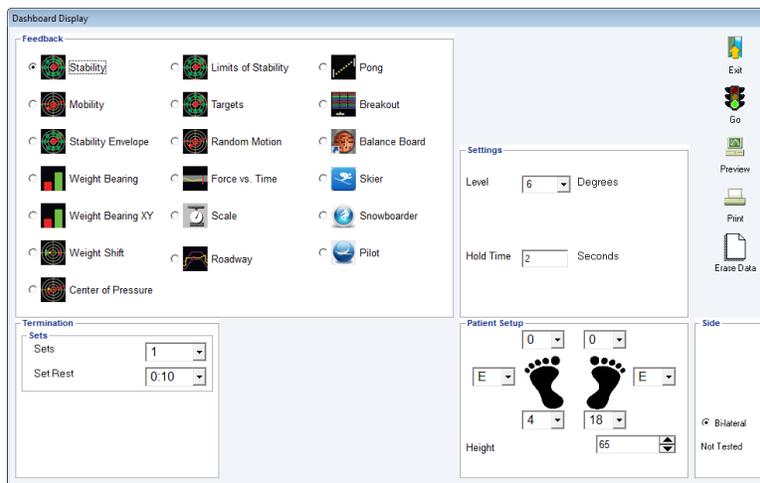


Figure 32 Dashboard Screen

Options

Status	Definition
Exit	Leave the Dashboard
Go	Start the selected Exercise
Preview	Preview a report from the recently completed exercise.
Print	Print a report from the recently completed exercise
Erase Data	Delete the data from the recently completed exercise

Side

The Side is used when printing reports. When you select a Feedback Display, the FallTrak II displays Right/Left or Bi-Lateral as the side. Certain displays e.g. Force vs. Time allow Uni-lateral testing. In this case you can select whether the Right or Left side is being exercised and the data will be correctly reported.

The FallTrak II displays the Status (Tested, No-Tested) indicating if there is data to be printed.

GROUP SUMMARY

The FallTrak II Group Summary program computes summary data over various patient populations. Summaries are stored in a Group Summary database, allowing the clinic to build a library of summary data results. Reports can be printed for a stored group or comparing a patient to the group.

Computing a Group Summary

1. From the **Group Summary** menu, select **My Data**.


Adding a New Group Summary to the Database

Steps

1. Select the **Pattern**, e.g. Standing Balance - Bilateral.
2. Select the **Protocol**.
3. Click the **Add** button.
4. Enter the **Summary name**, e.g. CTSIB and click the OK button.
5. The FallTrak II will add the new summary to the Group Summary Database.

Note: The FallTrak II has added a blank group summary record for the selected pattern and protocol. When you click the Summarize button, the FallTrak II reads through the database of tests and populates the group summary record.

Screen

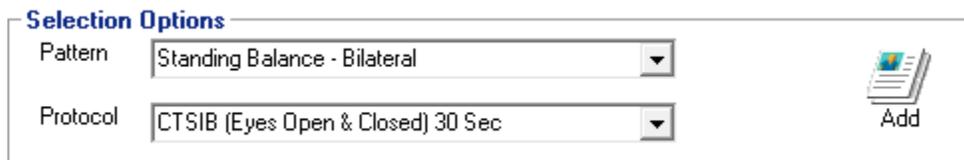

The screenshot shows a dialog box titled "Selection Options" with two dropdown menus. The first dropdown is labeled "Pattern" and has "Standing Balance - Bilateral" selected. The second dropdown is labeled "Protocol" and has "CTSIB (Eyes Open & Closed) 30 Sec" selected. To the right of the dropdowns is an "Add" button with a document icon.

Figure 33 Group Summary: Add

Computing the Summary

Setting the Group Criteria

The Group Summary filter allows you to select which patients will be included in the summary. The following example includes patients from 25 to 40 years old when the test was performed with an ACL Tear. Only tests which were performed within 31 days of the injury will be included. You can independently specify the From and To values, e.g.:

From	To	Data
Blank	Blank	Ignore the field
Blank	40	All patients <= 40 years old when the test was performed.
25	Blank	All patients >= 25 years old when the test was performed.
25	40	All patients from 25 to 40 years old when the test was performed.

Steps

1. Click the **Background** button.
2. Enter the **criteria**.
3. Click the **OK** button to save the criteria.

Screen

Figure 34 Group Summary: Filter

Select Tests

Figure 35 Group Summary: Select Tests

Options	Definition
All Tests	Include all tests for each patient in the summary.
Most Recent Test	Include the most recent test for each patient in the summary. This option prevents a patient who has been tested multiple times from skewing the results.

Side

Figure 36 Group Summary: Side

Side	Definition
Right vs. Left	Average Right side data vs. Left side data.
Dominant vs. Non-Dominant	Average Dominant side data vs. Non-Dominant side data.
Involved vs. UnInvolved	Average Involved side data vs. UnInvolved side data.

Summarizing the Data

Click the Summarize button. The FallTrak II will read each test in the database, compute the measured parameters, and store them in the Summary Database.

Note:

1. When you change the summary criteria, you must re-summarize the data to compute the results for the new group.
2. When you test additional patients which you want to include in the Group Summary, simply select the Stored Summary and click the **Summarize** button. The FallTrak II will re-compute the summary for all tests stored in the database.

Exporting a Summary to Microsoft Access

1. From the Group Summary database area, select the **Group Summary record** you want to Export.
2. Click the **Access** button.
3. The FallTrak II will export the data computed in the Summary to the FallTrak II **Results2009.MDB** file. Data from the Results database can be imported to other applications, e.g. SPSS.

Group Summary - Access Data Format

The Group Summary program stores the summarized data in the **Results2009.MDB** database file. The file is in the Access 2000 format.

Results.MDB includes a collection of tables containing the more detailed Patient Information, Dynamometer Settings, and Per-repetition values. Advanced users may want to use the Linked Table to perform advanced Queries and Data Analysis.

Tables

Table	Definition
Protocol	Table containing one row for each patient test or exercise session.
ProtocolRow	Table containing the settings for each set (row) in the Protocol.
Demographics	The patient background information for each Protocol.
Vals	The per-repetition values for each row in the ProtocolRow.

Fields

Each table include a description of its data fields. To access the field descriptions:

1. Open the **Results2015.MDB** file with Access.
2. Single-click the **table** to select it.
3. Click the **Design** icon.
4. The **Description** column contains the field description.

Printing a Summary Report

1. In the **Group Summary database** area, select the Group Summary record you want to Preview/Print.
2. Click the **Preview** or **Print** button.

Comparing a Patient to a Group Summary

1. From the Main FallTrak II menu, select the **Patient**.
2. From the **Group Summary** menu, select **My Data**.
3. In the **Group Summary Database** area, select the Group Summary record you want to compare the patient to.
4. In the **Patient** area, select the Patient session you want to compare to the Group Summary.
5. In the **Patient** area click the **Preview** or **Print** button.

Deleting a Group Summary

1. From the **Group Summary Database** area, select the Group Summary record you want to Delete.

2. Click the **Delete** button.

Note: *Deleting the Group Summary only deletes the summarized results. The original patient tests remain in the FallTrak II database.*

UTILITIES

Edit Background Information

The FallTrak II allows you to edit the patient background information for stored tests. This is typically used after you complete a test or exercise session and realize the patient's weight, injury, or other information was incorrect.

Steps

1. Select the Patient whose data you want to edit.
2. On the **Database** menu, click **Edit Background**.
3. Click the **Test/Exercise record** you want to edit.
4. Click **Update** to open the patient background information form.
5. Edit the patient information and click **OK** to save your changes.

Screen

Date	Pattern	Description	Test	Right	Left
4/27/2008 11:16:59 AM	Knee Extension/Flexion	3 Speed Protocol (60/180/240)	<input checked="" type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>
3/4/2008 10:27:27 AM	Knee Extension/Flexion	Resistance Sampling - CPM/Isom/Isoket	<input type="checkbox"/>	<input checked="" type="checkbox"/>	<input checked="" type="checkbox"/>
3/4/2008 10:09:39 AM	Knee Extension/Flexion	Interactive Path - Mid Range	<input checked="" type="checkbox"/>	<input checked="" type="checkbox"/>	<input checked="" type="checkbox"/>
3/4/2008 9:59:53 AM	Knee Extension/Flexion	Isometric 90/60/30 degrees	<input checked="" type="checkbox"/>	<input checked="" type="checkbox"/>	<input checked="" type="checkbox"/>
▶ 3/4/2008 9:51:17 AM	Knee Extension/Flexion	Isometric 90/60/30 degrees	<input checked="" type="checkbox"/>	<input checked="" type="checkbox"/>	<input checked="" type="checkbox"/>
8/9/2007 2:19:40 PM	Knee Extension/Flexion	60/180 Degrees Per Second	<input checked="" type="checkbox"/>	<input checked="" type="checkbox"/>	<input checked="" type="checkbox"/>
8/7/2007 8:27:12 AM	Knee Extension/Flexion	60/180 Degrees Per Second	<input checked="" type="checkbox"/>	<input checked="" type="checkbox"/>	<input checked="" type="checkbox"/>
8/5/2007 11:51:49 AM	Shoulder Internal/External Rotation, N	2 Speed Protocol (60/180)	<input checked="" type="checkbox"/>	<input checked="" type="checkbox"/>	<input checked="" type="checkbox"/>
8/5/2007 10:56:03 AM	Knee Extension/Flexion	Repeatability 60/90/60	<input checked="" type="checkbox"/>	<input checked="" type="checkbox"/>	<input checked="" type="checkbox"/>
8/4/2007 3:17:27 PM	Ankle Plantar/Dorsiflexion Prone	2 Speed Protocol (30/120)	<input checked="" type="checkbox"/>	<input checked="" type="checkbox"/>	<input checked="" type="checkbox"/>
8/4/2007 2:44:30 PM	Shoulder Internal/External Rotation, N	2 Speed Protocol (60/180)	<input checked="" type="checkbox"/>	<input checked="" type="checkbox"/>	<input checked="" type="checkbox"/>
8/4/2007 2:33:37 PM	Knee Extension/Flexion	60/180 Degrees Per Second	<input checked="" type="checkbox"/>	<input checked="" type="checkbox"/>	<input checked="" type="checkbox"/>

 Update

Figure 37 Edit Background Information

Deleting Stored Tests

The **Database, Delete Record** command allows you to delete stored test and exercise records from the database.

Steps

1. On the **Database** menu, click **Delete Record**.
2. Click the **Test/Exercise record** you want to delete.
3. Click **Delete** to delete the record.

Note

- Deleted records cannot be restored.
- Use the delete function in the patient selection form to quickly delete a patient and all their stored test and exercise sessions.

Screen

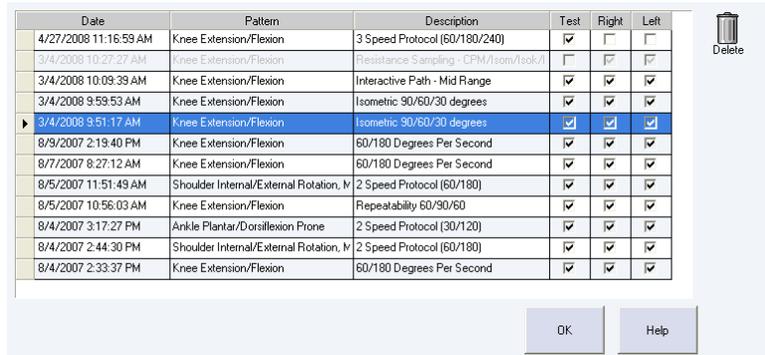


Figure 38 Delete Stored Test

Usage Report

The Usage Report shows each protocol run on the system grouped by patient. The report is sorted first by Patient Name, then by the Date the protocol was run.

Steps

1. On the **File** menu, click **Usage Report**.

Screen

HUMAC2008 Usage Report			
Date	Description	Protocol	Test
Potash, Rob L			
8/4/2007 2:33:37PM	Knee Extension/Flexion	60/180 Degrees Per Second	Yes
8/4/2007 2:44:30PM	Shoulder Internal/External Rotation, I	2 Speed Protocol (60/180)	Yes
8/4/2007 3:17:27PM	Ankle Plantar/Dorsiflexion Prone	2 Speed Protocol (30/120)	Yes
8/5/2007 10:56:03AM	Knee Extension/Flexion	Repeatability 60/90/60	Yes
8/5/2007 11:51:49AM	Shoulder Internal/External Rotation, I	2 Speed Protocol (60/180)	Yes
8/7/2007 8:27:12AM	Knee Extension/Flexion	60/180 Degrees Per Second	Yes
8/9/2007 2:19:40PM	Knee Extension/Flexion	60/180 Degrees Per Second	Yes
3/4/2008 9:51:17AM	Knee Extension/Flexion	Isometric 90/60/30 degrees	Yes
3/4/2008 9:59:53AM	Knee Extension/Flexion	Isometric 90/60/30 degrees	Yes
3/4/2008 10:09:39AM	Knee Extension/Flexion	Interactive Path - Mid Range	Yes
3/4/2008 10:27:27AM	Knee Extension/Flexion	Resistance Sampling - CPM/Isom/Isok/Is	No
4/27/2008 11:16:59AM	Knee Extension/Flexion	3 Speed Protocol (60/180/240)	Yes
Rudzitis, Paul V			
8/4/2007 2:16:54PM	Knee Extension/Flexion	60/180 Degrees Per Second	Yes
8/4/2007 2:55:02PM	Shoulder Internal/External Rotation, I	2 Speed Protocol (60/180)	Yes
8/4/2007 3:04:36PM	Ankle Plantar/Dorsiflexion Prone	2 Speed Protocol (30/120)	Yes
8/5/2007 11:07:18AM	Knee Extension/Flexion	Repeatability 60/90/60	Yes

Figure 39 Usage Report

Backup

We suggest you perform periodic backups to assure your data can be restored in the event of a computer failure. You can do a backup via an **External File Copy** or with the **FallTrak II Backup Command**.

Steps

2. On the **File** menu, click **Backup**.
3. Click **Set Backup Destination**.
4. From the Windows **File Open** dialog box, select the destination for the backup.
5. Select the **Files to Backup**. Typically you would select all files.
6. Click the **Backup** button.

Note Some sites prefer to use CD-R or CD-RW media for backup. These media require special programs provided with the CD drive to write data to the CD. If you will be using the FallTrak II Backup Command you must perform the backup to a folder on the computer's hard drive and then use the CD-supplied software to copy the FallTrak II Backup.ZIP file to the CD.

Screen

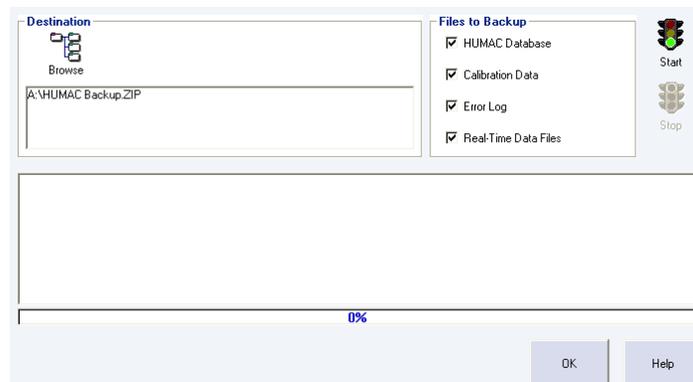


Figure 40 Backup Database

External File Copy

You can create a backup of your FallTrak II data by copying the FallTrak II data files to your backup media, e.g. CD-R or tape drive.

Options

Options	Definition
Set Backup Destination	During a backup the FallTrak II compresses your data into a file called FallTrak II Backup.ZIP . This file can be on floppy disk, a ZIP drive, a local hard drive (which should be different than the drive holding the FallTrak II database), or a network. The Set Backup Destination option allows you to specify where the backup file will be created.
Files to Backup	The Files to Backup area allows you to select which files will be added to your backup. Typically you would select all of the files. When contacting CSMI to resolve a question, we may request you only select one type of file for backup and then send the backup to CSMI.
Backup	Start the backup.
Abort	Abort the backup currently in progress and delete the backup file.

Data File Location

The FallTrak II database can be stored on your local computer or on a network.

Steps

1. On the **File** menu, click **Database, Open**.
2. When the Windows **Open File** dialog box appears, locate and open your FallTrak II database (**User FallTrak II.MDB**).

Compress

If you make frequent changes in the FallTrak II database, sections of the database may become fragmented. Therefore, it is a good idea to periodically compress the database.

Schedule the compression depending on how much the data changes. If the data does not change that often, you do not have to compress that much. If there are many and frequent updates, inserts, and deletes, compress more. Even though there is no formal rule for how often to compact, Microsoft recommends that you compress on a regular basis.

For a detailed description of the compress function, see the Microsoft Knowledge Base article [Q209769 ACC2000: Defragment and Compact Database to Improve Performance](#).

Steps

1. On the **File** menu, click **Compress**.
2. Click **Yes** to compress the database.

Note During the compress function the mouse pointer will change to an hourglass. Because the compress function is a Microsoft internal database function you will not see any progress indicators. The FallTrak II will display a message when the compression is successfully completed.

Import: Database

The FallTrak II **Import Database** command allows you to combine data from multiple FallTrak II databases into a single database for reporting and group summary analysis. Each FallTrak II Database is a unique GUID. Use the **Help, About** command to view the GUID for your database.

IMPORTANT: The Import Database function is used to combine multiple databases into a single FallTrak II Database at a central location. To upgrade an existing database to the FallTrak II2009 format, use the File, Upgrade command.

Steps

Importing Data When the Sending Sites Backup Their FallTrak II Data

1. Each site creates a backup of their data and sends it to the central location.
2. On the **File** menu, click **Import, Database**.
3. Click **Set Source**.
4. From the Windows **File Open** dialog box, select the FallTrak II Backup file submitted by the site.
5. Click **UnZip Data** to expand the FallTrak II Backup file.
6. Click **Import Data** to import the submitted data into the main FallTrak II database.

Importing Data When the Sending Sites Copy Their FallTrak II Data

1. Each site copies the FallTrak II Data Files to a CD-R, ZIP or other large-format storage media.
2. On the **File** menu, click **Import, Database**.
3. Click **Set Source**.
4. From the Windows **File Open** dialog box, select the **User FallTrak II.MDB** file submitted by the site.
5. Click **Import Data** to import the submitted data into the main FallTrak II database.

Screen

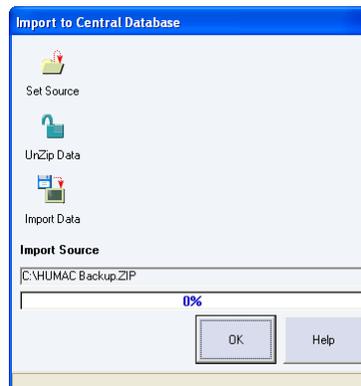


Figure 41 Import Database

Options

Options	Definition
Set Source	Point to the source media (CD, ZIP disk) containing the FallTrak II Backup data.
UnZip Data	UnZip a FallTrak II Backup file.
Import Data	Read the backup data into the database.

Import: Protocols

The FallTrak II **Import Protocols** command allows you to import System Protocols from other users into your FallTrak II System.

Steps

1. On the **File** menu, click **Import, Protocols**.
2. Each FallTrak II protocol has a unique code. The FallTrak II will ask if you want to **Overwrite duplicate protocols**, e.g. those with the same unique code.
 - a. Yes: Import the protocol, overwriting the existing system protocol with the imported protocol.
Note: No changes are made to tests which were previously run with the protocol.
 - b. No: Do not import the protocol.
3. From the Windows **File Open** dialog box, select the **Protocols2009.MDB** file and click the **Open** button.
4. The FallTrak II will import the System Protocols to your database.

Exporting Data

The FallTrak II allows you to Export real-time data to an Excel Spreadsheet or to a Text File.

Steps

1. Select the Patient whose data you want to export.
2. On the **Database** menu, click **Export Data**.
3. Click the **Test/Exercise record** you want to export.
4. Click **Excel** to export the data to an Excel Spreadsheet. Click **Text** to export the data to a text file.

Excel File Format

The FallTrak II will create an Excel Workbook containing one sheet for each test/exercise set. Each sheet will contain the real-time time, torque and position data along with a plot of the torque and position data vs. time.

Text File Format

The FallTrak II will create a text file for each test speed and side run. The files are placed in the FallTrak II/Export folder with the file name identifying the specific patient and test.

Patient Name, Date/Time of Test, Speed, Repetitions, Protocol Set Number, ".CSV"

The Protocol Set Number indicates the set in the protocol, because you can create a protocol with multiple sets of the same speed and number of repetitions. For example, if we tested Rob Potash on September 13 at 60 deg/sec 15 repetitions and 180 deg/sec 15 repetitions, the FallTrak II would create the following files.

```
Potash_ Robert L 9_12_98 7_48_40 AM 60 5 Right 0.CSV  
Potash_ Robert L 9_12_98 7_48_40 AM 60 5 Left 0.CSV  
Potash_ Robert L 9_12_98 7_48_40 AM 180 15 Right 1.CSV  
Potash_ Robert L 9_12_98 7_48_40 AM 180 15 Left 1.CSV
```

These tests can be read into Excel, by double-clicking the filename.

Submitting a Test to MedTrak VNG

The FallTrak II allows you to submit tests to MedTrak VNG via for review.

Steps

1. Select the **Patient** whose data you want to submit.
2. On the **Database** menu, click **Database, Submit Test**.
3. Click the **Browse** button to set the destination for the submitted test file. If you do not have internet access from the computer, you can set the destination to a USB key and e-mail the test to MedTrak VNG from a different computer.
4. Select the **Test** or **Exercise** you want to submit.
5. Click the **Start** button to create the Submit file.
6. E-mail the file to scott@medtrakvng.com. Add a description of the issue you would like MedTrak VNG to review in the body of the e-mail when submitting the test.

Note: *Patient identifying information (Name, ID, Doctor, etc.) is removed from the test before it is submitted to MedTrak VNG. In addition the Submit file is encrypted and password protected.*

System Protocols

The FallTrak II allows you to define System Protocols. System Protocols can be selected for use by any patient.

Screen

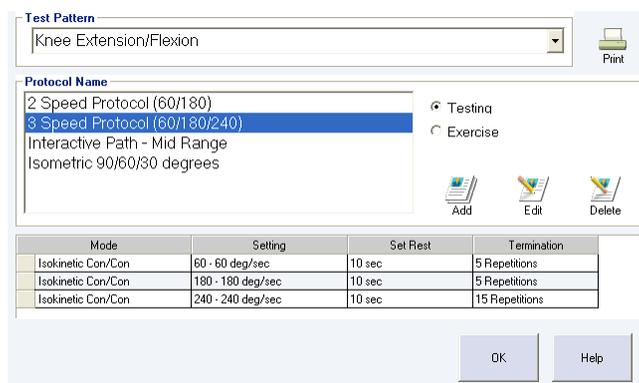


Figure 42 System Protocol Editor

Editing a System Protocol

1. From the **Utilities** menu, select **Protocol Editor**.
2. In the **Test Pattern** area, select the pattern you want to add/edit a protocol for. The FallTrak II will display all available system protocols for the selected Pattern in the Protocol Name area.
3. Select **Testing** or **Exercise** to edit the appropriate protocol.
4. Single-click the protocol name. The FallTrak II will display all sets in the selected protocol.
5. Select **Edit** to edit the selected protocol.

Adding a System Protocol

1. From the **Utilities** menu, select **Protocol Editor**.
2. In the **Test Pattern** area, select the pattern you want to add/edit a protocol for. The FallTrak II will display all available system protocols for the selected Pattern in the Protocol Name area.
3. Select **Testing** or **Exercise** to edit the appropriate protocol.
4. Select **Add** to add a new protocol.

Deleting a System Protocol

1. From the **Utilities** menu, select **Protocol Editor**.
2. In the **Test Pattern** area, select the pattern you want to add/edit a protocol for. The FallTrak II will display all available system protocols for the selected Pattern in the Protocol Name area.
3. Select **Testing** or **Exercise** to edit the appropriate protocol.
4. Single-click the protocol name. The FallTrak II will display all sets in the selected protocol.
5. Select **Delete** to delete the selected protocol.

Printing Protocols

1. From the **Utilities** menu, select **Protocol Editor**.
2. Click the **Print** button.

Interactive Protocols

The FallTrak II allows you to define a number of interactive protocols. The interactive protocols are described in detail below. The procedures for adding, editing and deleting interactive protocols are similar for each protocol type.

Adding a New Protocol

1. From the **Utilities** menu, select **Interactive Feedback Editor, Protocol Type**.
2. Click the **Add** button.
3. Enter the protocol values. The FallTrak II automatically updates the feedback display.
4. In the area above the display enter a **Descriptive Name** for the protocol.

Editing a Protocol

1. From the **Utilities** menu, select **Interactive Feedback Editor, Protocol Type**.
2. In the table on the left, click the **Protocol Name** you want to edit.
3. Enter the protocol values. The FallTrak II automatically updates the feedback display.
4. In the area above the display enter a **Descriptive Name** for the protocol.

Deleting a Protocol

1. From the **Utilities** menu, select **Interactive Feedback Editor, Protocol Type**.
2. In the table on the left, click the **Protocol Name** you want to delete.
3. Click the **Delete** button.

Protocol Editor: Roadway

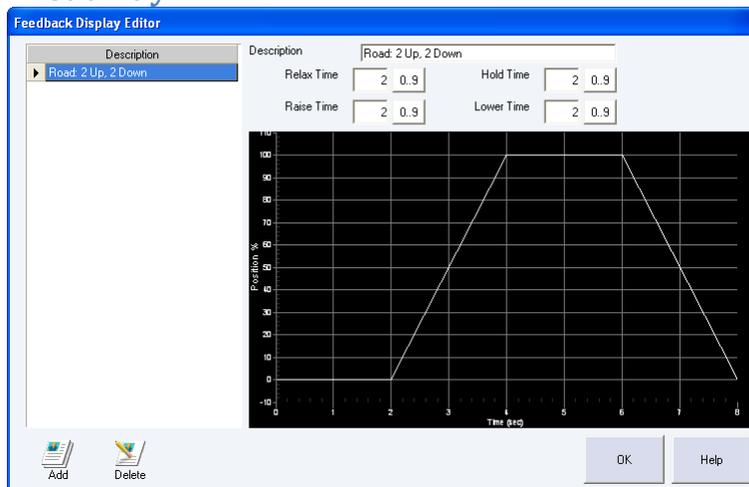


Figure 43 Protocol Editor: Roadway

Parameter	Definition (in seconds)
Relax Time	Time during which the patient relaxes at the start of a repetition.
Raise Time	Time during which the patient is moving toward the ROM target in Isotonic mode or the Torque target in Isometric mode.
Hold Time	Time during which the patient holds at the ROM target in Isotonic mode or the Torque target in Isometric mode.
Lower Time	Time during which the patient is relaxing and moving toward the starting position in Isotonic mode or not generating torque in Isometric mode.

FallTrak II Data File Locations

Files

The FallTrak II uses an Access Databases and a collection of real-time data files to store the Balance data.

FileName	Description
User FallTrak II.MDB	Patient names, demographic information, dynamometer settings, protocols, and pointers to the real-time data files.
User Calibration2015.MDB	Dynamometer calibration and verification data.
User Error Log2015.MDB	Log of Errors reported by the FallTrak II System.
Results2015.MDB	Exported Group Summary data.
*.H09	Real-time data files containing the torque and position data collected during each test and exercise set.
Letter.Doc	Narrative Report merge document.
Letter-IsoM.Doc	Isometric Narrative Report merge document.

File Locations

Windows8, Windows7, Vista

FileName	Location
MDB Files	C:\Users\Public\Public Documents\MedTrak VNG\FallTrak II
*.H09 files	C:\Users\Public\Public Documents\MedTrak VNG\FallTrak II\Data and all sub-folders.
Letter*.Doc	C:\Users\Public\Public Documents\MedTrak VNG\FallTrak II\Letter
Exported Text Files	C:\Users\Public\Public Documents\MedTrak VNG\FallTrak II\Export

WindowsXP

FileName	Location
MDB Files	C:\Documents and Settings\All Users\Shared Documents\MedTrak VNG\FallTrak II
*.H09 files	C:\Documents and Settings\All Users\Shared Documents\MedTrak VNG\FallTrak II\Data and all sub-folders.
Letter*.Doc	C:\Documents and Settings\All Users\Shared Documents\MedTrak VNG\FallTrak II\Letter
Exported Text Files	C:\Documents and Settings\All Users\Shared Documents\MedTrak VNG\FallTrak II\Export

Windows98

FileName	Location
MDB Files	C:\Program Files\MedTrak VNG\FallTrak II
*.H09 files	C:\Program Files\MedTrak VNG\FallTrak II\Data and all sub-folders.
Letter*.Doc	C:\Program Files\MedTrak VNG\FallTrak II\Letter
Exported Text Files	C:\Program Files\MedTrak VNG\FallTrak II\Export

Users

The FallTrak II program allows you to setup user accounts which include passwords and administrative privileges.

Steps

1. From the **File** menu, select **Setup Users, Setup**.

Adding a New User

1. Click the **Add** button.

Field	Description
Display Name	The therapist's name recorded in the access log.
Username	The therapist's login name.
Password	The therapist's login password. Note: Passwords are case sensitive.
Administrator	Only administrators can access the File, Setup Users function.

Editing an Existing User

1. Select the **user**.
2. Click the **Edit** button.

Deleting an Existing User

1. Select the user.
2. Click the **Delete** button.

Screen

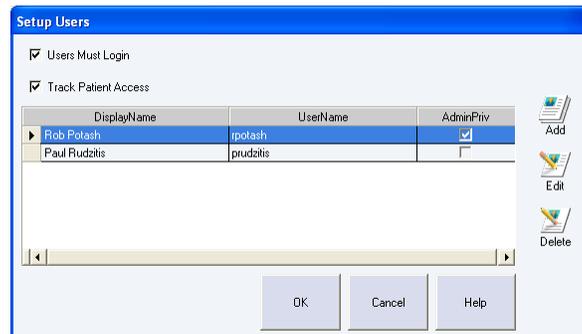


Figure 44 Setup Users

Options

Field	Description
Users Must Login	When the FallTrak II starts, the user must enter their username and password.
Track Patient Access	Record the therapist name, patient name and date/time when a patient is selected.
Add	Add a new user
Edit	Edit the selected user's record.
Delete	Delete the selected user. Note: Deleting the selected user removes them from the Login options. All recorded database access is retained after the user is deleted.

The selected user is deleted from the database. No delete confirmation is requested. **Note:** Deleting the selected user removes them from the Login options. All recorded database access is retained after the user is deleted.

Login

Steps

1. If **Users Must Login** is selected, the FallTrak II will display the **Login** screen at startup.

2. Use the **File, Setup Users, Login** command to change the logged-in user without re-starting the FallTrak II,

Screen

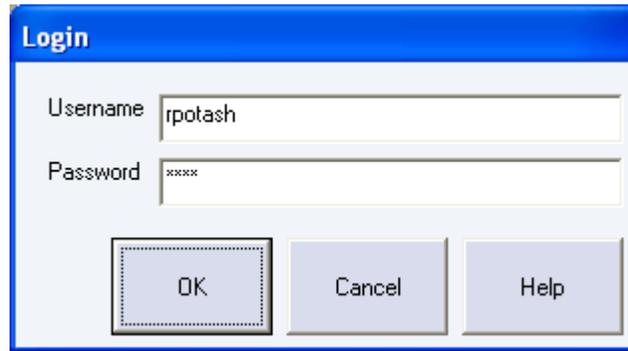


Figure 45 Login

Printing an Access Log

1. From the **File** menu, select **Report, Access Log**.

PREFERENCES

The FallTrak II **File, Preferences** command allows you to customize the operation of your FallTrak II System. The Preferences dialog box is divided into four tabs: **General, Data Analysis, Reporting,** and **Balance**.

Language

The FallTrak II is available in a number of languages. To set the language:

Steps

1. On the **File** menu, click **Preferences**.
2. Click the **General** tab.
3. Select the **Language**.
4. Click the **OK** button.
5. Exit and re-start the FallTrak II program to switch to the new language.

Note: *The Windows Operating System must include the fonts required to display the selected language. Selecting Japanese on an English version of Windows will display ??? for the text as the Japanese fonts are not available on English Windows systems.*

Touch Screen Display

The FallTrak II can increase the grid sizing for use with touch screen monitors.

Steps

1. On the **File** menu, click **Preferences**.
2. Click the **General** tab.
3. Set the **Touch Screen** option.

FallTrak II Interface Type

Selects the type of interface (Demo, USB Port) used to connect to the FallTrak II.

Steps

1. On the **File** menu, click **Preferences**.
2. Click the **General** tab.
3. Select the **Interface**.

Options

Option	Description
None, run program in Demo mode	This is used if you want to evaluate the FallTrak II program before installation on a MedTrak VNG. The FallTrak II will simulate the Balance System allowing you to test all FallTrak II functions.
USBCOMM	Connect to the Balance Platform.

Use Sound to Count Repetitions

The FallTrak II can count repetitions through the PC's speakers during testing and exercise.

Steps

1. On the **File** menu, click **Preferences**.
2. Click the **Feedback** tab.
3. Click on **Use sound to count repetitions**.

Reset Torque Scales for Each Set

The FallTrak II auto-scales the feedback display during testing and exercise. The very first time a patient is tested, the FallTrak II starts with a 20 ft-lbs scale. If the patient exceeds 20 ft-lbs, the FallTrak II moves to a 50 ft-lbs scale. The FallTrak II continues with 100, 200, and 500 ft-lbs scales until the best scale for the patient is found. This best value is stored in the FallTrak II database. The next time the patient is tested, the FallTrak II starts with best value.

Because of the potential strength difference between involved and un-involved limbs and different test speeds in a protocol, some clinicians prefer to have the FallTrak II find the best torque scale for each set.

Steps

1. On the **File** menu, click **Preferences**.
2. Click the **Feedback** tab.
3. Check **Reset torque scales for each set** to have the FallTrak II find the best torque scale for each set. Clear **Reset torque scales for each set** to have the FallTrak II find the best torque scale for the patient and apply it to all sets.

Report Heading

The FallTrak II allows you to specify a heading (typically your clinic name and address) to be printed at the top of each report.

Steps

1. On the **File** menu, click **Preferences**.
2. Click the **Reporting** tab.
3. Enter the **Report Heading**.

Units

The FallTrak II supports US and Metric units.

Steps

1. On the **File** menu, click **Preferences**.
2. Click the **Reporting** tab.
3. In the **Units** section, select **US Units** or **Metric Units**.

Options

Data	US	Metric
Height	inches	centimeters
Weight	pounds	kilograms
Torque Limit	Foot-pounds	newton-meters
Torque	Foot-pounds	newton-meters
Work	Foot-pounds	newton-meters

Limits of Stability Normative Values

The FallTrak II allows you to specify normative values for the Limits of Stability test.

Steps

1. On the **File** menu, click **Preferences**.
2. Click the **Balance** tab.
3. Enter the Normative Values for each target. **Note:** *The center target is the Normative Value for the Overall Score.*

Stability Hold Time

The FallTrak II allows you to specify default **Hold Time** for the **Stability Test** used in the **Dashboard**.

Steps

1. On the **File** menu, click **Preferences**.
2. Click the **Balance** tab.
3. Enter the **Hold Time** value.

Stability Goal

The FallTrak II allows you to specify the Normative Value for the **Stability Goal**.

Steps

1. On the **File** menu, click **Preferences**.
2. Click the **Balance** tab.
3. Enter the **Stability Goal** value.

Weight Bearing Display

The FallTrak II allows you to specify the Percent Right/Left or the Force Right/Left for the Weight-Bearing display.

Steps


1. On the **File** menu, click **Preferences**.
2. Click the **Balance** tab.
3. In the **Weight Bearing** area, select **Display %** or **Display Weight**.

Measured Parameters

Sensor Layout

Force Platform Mode

The balance sensors are arranged as follows:

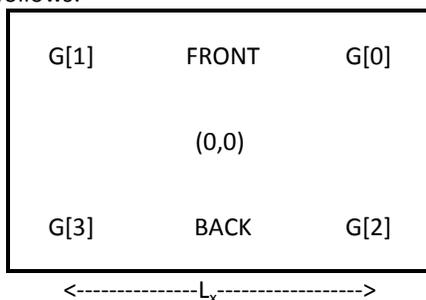

The board dimensions are defined as

Parameter	Definition
Lx	Distance from G[1] to G[0]
Ly	Distance from G[1] to G[3]

Tilt Mode

The Tilt Sensor is a single device which provides tilt measurement in the X and Y axes.

Patient Center of Mass (Force Platform Mode)

The Center of Mass for a patient is defined as 55% of their height.

Weight (Force Platform Mode)

The patient weight is computed as the sum of the four gauges

$$WEIGHT = \sum_{n=0}^3 G[n]$$


Center of Pressure – Balance Platform (Force Platform Mode)

The Center of Pressure on the board is computed as follows:

$$PLATFORM_COP_x = \left(\frac{G[0] + G[2]}{WEIGHT} \right) * L_x - \frac{L_x}{2}$$

$$PLATFORM_COP_y = \left(\frac{G[0] + G[1]}{WEIGHT} \right) * L_y - \frac{L_y}{2}$$

Foot Position

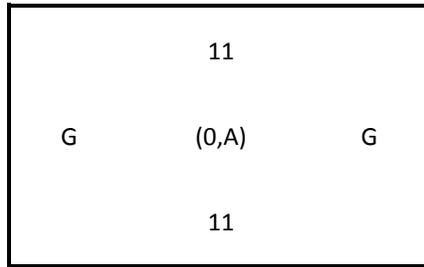

The platform is arranged with 0.75" grid spacing.

X-Axis locations are specified as a letter running from "A" at the center of the board to "G" at each edge of the board.

Y-Axis locations are specified as a number running from 0 at the center of the board to 11 at each edge of the board.

The Foot Offset (in inches from (0,0) at the center of the board) is computed as follows:

$$FOOT_OFFSET_x = (FOOT_LOCATION_x - 11) * GRID_SPACING$$

$$FOOT_OFFSET_y = (ASC(GRID_LOCATION_y) - ASC("G")) * GRID_SPACING$$

Note: ASC is the computer function which converts the letter to a numeric code so:

$$ASC("G") - ASC("E") = 2$$

Center of Pressure - Patient (Force Platform Mode)

We need to adjust the measured COP relative to the patient's COP, not the Platform's COP. In this way when the patient is balanced the FallTrak II will display the COP at (0,0) on the screen. We do this one of two ways:

1. Measure the Gauge Output when the therapist says the patient is in a neutral (Anatomical Zero) position.
2. Compute the AZ based on the Patient's Foot Offset from the board COP.

Anatomical Zero Based

If the therapist sets a Patient AZ, the FallTrak II uses the AZ to compute the patient Center of Pressure. The FallTrak II assumes the feet are equally-spaced along the X-Axis from the centerline of the board.

$$PATIENT_COP_x = PLATFORM_COP_x - PATIENT_AZ_COP_x$$

$$PATIENT_COP_y = PLATFORM_COP_y - PATIENT_AZ_COP_y$$

Foot Position Based

If an AZ is not set, the FallTrak II uses the patient foot position to compute the Center of Pressure. The FallTrak II computes the Foot Offset for the Y-Axis as the average offset from (0,0) of the patient's Right and Left side mandibular bones.

Axis	Location
X	Midway between their feet
Y	In-line with their mandibular bone

$$FOOT_OFFSET_x = 0$$


$$FOOT_OFFSET_Y = \left(\frac{FOOT_OFFSET_{RIGHT} + FOOT_OFFSET_{LEFT}}{2} \right)$$

$$PATIENT_COP_X = PLATFORM_COP_X$$

$$PATIENT_COP_Y = PLATFORM_COP_Y - FOOT_OFFSET_Y$$

The Patient Neutral COP is the COP where they feel they are standing in a centered position.

COP Length, Angle, Direction (Force Platform Mode)

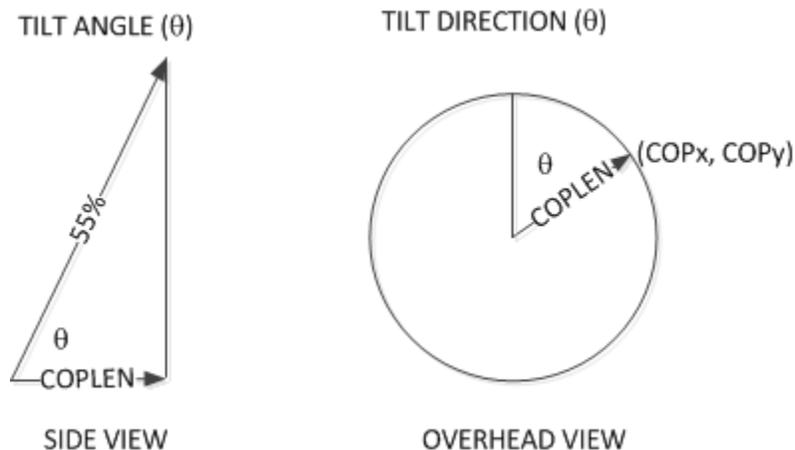

The COP Length is computed as:

$$COP_LENGTH = \sqrt{COP_X^2 + COP_Y^2}$$

The Patient's Tilt Angle is computed as:

$$TILT_ANGLE = \cos^{-1} \left(\frac{COPLEN}{PATIENT_HEIGHT * 0.55} \right)$$

The Patient's Tilt Direction is computed as:

$$TILT_DIRECTION = \tan^{-1} \left(\frac{COP_X}{COP_Y} \right)$$

Displayed Tilt Angle, Tilt Direction (Force Platform Mode)

The displayed COP is computed as a projection of the Tilt Angle and Direction onto the platform. For example if the patient was tilted 6 degrees directly to the right, the FallTrak II would display the cursor on the 6 degree circle at 3:00.

$$PROJECTED_COP_X = TILT_ANGLE * \sin(TILT_DIRECTION)$$

$$PROJECTED_COP_Y = TILT_ANGLE * \cos(TILT_DIRECTION)$$

Displayed R/L and A/P Tilt Angle (Tilt Mode)

The displayed Right/Left and Anterior/Posterior Tilt Angles are read directly from the Tilt Sensor and indicate the Balance Platform angle.

$$PROJECTED_COP_x = TILT_ANGLE_{R/L}$$

$$PROJECTED_COP_y = TILT_ANGLE_{A/P}$$

Percent Weight Bearing (Force Platform Mode)

Right/Left

The Right/Left Weight Bearing values are computed as:

$$PCT_RIGHT = ABS\left(\frac{FOOT_OFFSET_LEFT_x - COP_x}{FOOT_OFFSET_LEFT_x - FOOT_OFFSET_RIGHT_x}\right) * 100$$

$$PCT_LEFT = 100 - PCT_RIGHT$$

The value is limited to 100%. Since the centerline of the patient's foot is recorded, it is possible they can shift all of their weight to the outer edge of their right foot which would cause the PCT_RIGHT to be greater than 100% since the COP_x would be outside the FOOT_OFFSET_RIGHT.

Front/Back

The Front/Back Weight Bearing values are computed as:


$$PCT_FRONT = ABS\left(\frac{FOOT_OFFSET_LEFT_y - COP_y}{FOOT_OFFSET_LEFT_y - FOOT_OFFSET_RIGHT_y}\right) * 100$$

$$PCT_BACK = 100 - PCT_FRONT$$

Note: Since the patient is not shifting from one foot to the other in the Y-Axis there is no true percent weight bearing. The FallTrak II uses the Right/Left foot positions for these calculations so an equivalent shift Front/Back in the COP results in the same reported percentage as the Right/Left.

Limits of Stability Score (Force Platform Mode and Tilt Mode)

The ratio of the length a straight line between to targets to the length of the path traversed by the patient between the targets.

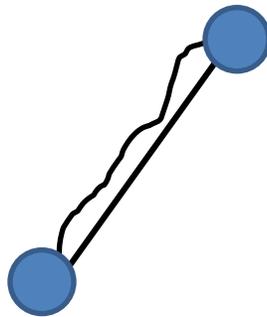

$$LOSSCORE = \left(\frac{LENGTH_STRAIGHT}{LENGTH_PATIENT}\right) * 100$$

Stability Score (Force Platform Mode and Tilt Mode)

The ratio of the percent of time the patient was on a target to the time the target was active.

$$STABILITY_SCORE = \left(\frac{TIME_ON_TARGET}{TIME_TARGET_ACTIVE}\right) * 100$$

Mobility Score (Force Platform Mode and Tilt Mode)

The ratio of the percent of time the patient was on the target to the duration of the test.

$$MOBILITY_SCORE = \left(\frac{TIME_ON_TARGET}{TOTAL_TEST_TIME} \right) * 100$$

CTSIB Score (Force Platform Mode)

The CTSIB Score is computed as the ratio between the Patient's 95% confidence interval major axis length and the standard 6.25 degree tilt axis length.

The COP_LENGTH is defined as:

$$RADIUS = \sqrt{COP_x^2 + COP_y^2}$$

Compute the COP_LENGTH for 6.25 degrees of tilt.

$$COP_LENGTH_{6.25} = 0.55 * PATIENTHEIGHT * \sin(6.25)$$

Compute the 95% confidence interval MAJOR_AXIS. **Note:** The 95% Confidence Interval calculations are complex and not included in the User Manual. The equations can be found on a number of websites.

Compute the STABILITY_SCORE as the ratio between the Patient's MAJOR_AXIS length and the 6.25° COP_LENGTH.

$$STABILITY_SCORE = \left(\frac{COP_LEN_{6.25} - 95\% \text{ CONFIDENCE INTERVAL}_{MAJOR_AXIS}}{COP_LEN_{6.25}} \right) * 100$$

Excel Export Fields

The following fields are exported to Excel.

Field	Units	Definition
Status		Program Status (See below)
Time	Seconds	Elapsed Time
Gauge[0..3]	Pounds	The Force on each strain gauge
Platform (X, Y)	Inches	The Center of Pressure on the Balance Platform where (0,0) is the center of the platform.
Weight	Pounds	The patient's weight.
Patient COP (X,Y)	Inches	The patient's Center of Pressure in inches where (0,0) is the patient's center of balance.
Patient COP Len	Inches	The length in inches of the patient's COP.
Tilt Angle	Degrees	The patient's Tilt Angle in Degrees
Tilt Direction	Degrees	The Direction of the patient Tilt Angle where 0 is 12:00 with values increasing in a clockwise direction.
Tilt Projection (X, Y)	Degrees	The patients Tilt Angle and Direction projection in degrees where (0,0) is the patient's center of balance.
Pct Right	%	The Percent of Body Weight on the Right foot.
Pct Front	%	The percentage of Body Weight in the Forward direction.

Program Status fields.

Value (HEX)	Definition
0020	Isometric mode: Patient not in target area, display frozen.

0040	Next external stimulus is presented to patient. First point when the target moved to the new location.
0080	The challenge has begun. Used to ignore data at the front of a test where the patient is getting ready to start.
0100	Patient is within the target area.
0200	Patient was on target the required amount of time so we can move-on to the next target.
0400	Patient was unable to reach the target so therapist moved on to the next target.
7800	Four bits used to indicate active target number (0-15).
8000	Patient fell during the test.

ELECTROMAGNETIC COMPATIBILITY

Table 1

Guidance and Manufacturer's Declaration – Emissions Guidance and Manufacturer's Declaration – Electromagnetic Emissions

The FallTrak II Balance is intended for use in the electromagnetic environment specified below. The customer or user of the FallTrak II should ensure that it is used in such an environment.

Emissions Test	Compliance	Electromagnetic Environment – Guidance
RF Emissions CISPR 11	Group 1	The FallTrak II uses RF energy only for its internal function. Therefore, its RF emissions are very low and are not likely to cause any interference in nearby electronic equipment.
RF Emissions CISPR 11	Class A or B	B
Harmonics IEC 61000-3-2	Class A,B,C,D or N/A	A
Flicker IEC 61000-3-3	Complies or N/A	Complies
	[See 6.8.3.201 a) 3) and Figure 201]	The FallTrak II is suitable for use in all establishments, including domestic, and those directly connected to the public low-voltage power supply network that supplies buildings used for domestic purposes.
RF Emissions CISPR 14-1	Complies	The FallTrak II is not suitable for interconnection with other equipment.
RF Emissions CISPR 15	Complies	The FallTrak II is not suitable for interconnection with other equipment.

Table 2

Guidance and Manufacturer’s Declaration – Immunity All Equipment and Systems Guidance and Manufacturer’s Declaration – Electromagnetic Immunity

The FallTrak II is intended for use in the electromagnetic environment specified below. The customer or user of the FallTrak II should ensure that it is used in such an environment.

Immunity Test	EN/IEC 60601 Test Level	Compliance Level	Electromagnetic Environment – Guidance
ESD EN/IEC 61000-4-2	±6kV Contact ±8kV Air	As specified	Floors should be wood, concrete or ceramic tile. If floors are synthetic, the r/h should be at least 30%
EFT EN/IEC 61000-4-4	±2kV Mains ±1kV I/Os	As specified	Mains power quality should be that of a typical commercial or hospital environment.
Surge EN/IEC 61000-4-5	±1kV Differential ±2kV Common	As specified	Mains power quality should be that of a typical commercial or hospital environment.
Voltage Dips/Dropout EN/IEC 61000-4-11	>95% Dip for 0.5 Cycle 60% Dip for 5 Cycles 30% Dip for 25 Cycles >95% Dip for 5 seconds	As specified As specified As specified	Mains power quality should be that of a typical commercial or hospital environment. If the user of the FallTrak II expects dips as specified and the FallTrak II requires continued operation during power mains interruptions, it is recommended that the FallTrak II be powered from an uninterruptible power supply or battery.
Power Frequency 50/60Hz EN/IEC 61000-4-8	3A/m	As specified	Power frequency magnetic fields should be that of a typical commercial or hospital environment.

Table 3

Guidance and Manufacturer’s Declaration – Emissions Equipment and Systems that are NOT Life-supporting Guidance and Manufacturer’s Declaration – Electromagnetic Immunity

The FallTrak II is intended for use in the electromagnetic environment specified below. The customer or user of the FallTrak II should ensure that it is used in such an environment.

Immunity Test	EN/IEC 60601 Test Level	Compliance Level	Electromagnetic Environment – Guidance
Conducted RF EN/IEC 61000-4-6	3 Vrms 150 kHz to 80 MHz	(V1)Vrms	Portable and mobile communications equipment should be separated from the FallTrak II360 by no less than the distances calculated/listed below: D=(3.5/V1)(Sqrt P) D=(3.5/E1)(Sqrt P) 80 to 800 MHz D=(7/E1)(Sqrt P) 800 MHz to 2.5 GHz where P is the max power in watts and D is the recommended separation distance in meters. Field strengths from fixed transmitters, as determined by an electromagnetic site survey, should be less than the compliance levels (V1 and E1). Interference may occur in the vicinity of equipment containing a transmitter.
Radiated RF EN/IEC 61000-4-3	3V/m 80 MHz to 2.5 GHz	(E1)V/m	

Table 4

Recommended Separation Distances between portable and mobile RF Communications equipment and the FallTrak II Balance Equipment and Systems that are NOT Life-supporting

The FallTrak II is intended for use in the electromagnetic environment in which radiated disturbances are controlled. The customer or user of the FallTrak II can help prevent electromagnetic interference by maintaining a minimum distance between portable and mobile RF Communications Equipment and the FallTrak II as recommended below, according to the maximum output power of the communications equipment.

Max Output Power (Watts)	Separation (m) 150kHz to 80MHz $D=(3.5/\sqrt{P})(\sqrt{P})$	Separation (m) 80 to 800MHz $D=(3.5/\sqrt{E1})(\sqrt{P})$	Separation (m) 800MHz to 2.5GHz $D=(7/\sqrt{E1})(\sqrt{P})$
0.01	.1166	.1166	.2333
0.1	.3689	.3689	.7378
1	1.1666	1.1666	2.3333
10	3.6893	3.6893	7.3786
100	11.6666	11.6666	23.3333